

CAREX[®]

ITEM# FGB30600 0000 + FGB30700 0000

Toilet Seat Elevator



 **Please read all instructions before assembly or use.**

Questions, problems or missing parts?

Before returning, contact Compass Health Brands customer service:

800-526-8051 | CAREX.COM

! WARNINGS

INDICATIONS FOR USE:

A Toilet Seat Elevator is intended to increase the height of the seating surface and can be used by persons with decreased strength or mobility. This device is intended for a single user **ONLY**.

CONTRAINDICATIONS:

For safety reasons, people with balance disorders should **NOT** remain without supervision when using this device. User **MUST** be able to maintain even weight distribution while lowering and rising from device and while seated. This device **MUST NOT** be used by individuals whose feet **DO NOT** touch the floor when seated, as appropriate balance on the device will not be possible.

SAVE THESE INSTRUCTIONS. This sheet MUST accompany the device with each use.

! WARNINGS:

• **DO NOT ATTEMPT TO ADJUST OR OPERATE THE TOILET SEAT ELEVATOR WITHOUT READING ALL INSTRUCTIONS CAREFULLY.**

- Inspect all parts for shipping damages before assembly. If there is shipping damage – **DO NOT USE**. Contact your dealer/carrier for further instructions.
- If you **DO NOT** understand these instructions, contact a healthcare dealer or technical professional before attempting to install or use the device.
- **WARNING:** Users that are unable to understand or follow these instructions and warnings or have limited physical capabilities should be supervised or assisted while using the Toilet Seat Elevator.
- **WARNING:** This Toilet Seat Elevator has a weight limit of 300 lbs (136 kg). **ALWAYS** observe the weight limit on the labeling of your Toilet Seat Elevator.
- Use the Toilet Seat Elevator only for the purpose for which it was designed. Compass Health Brands is **NOT** responsible in any way for any misuse of the Toilet Seat Elevator, or for injuries or damage caused as a result of it.
- Check that all labels are present and legible. Replace if necessary.
- **WARNING: DO NOT** use this device if your feet cannot make contact with the floor when in a seated position, as this will change the weight distribution and stability of the device, which can result in serious injury.
- It is recommended that a permanently installed wall-mount grab bar is used in addition to the Toilet Seat Elevator. Discuss your needs with your health care provider.
- **DO NOT** use the Toilet Seat Elevator if wobbly, unstable, or not level. **ALWAYS** be sure that the seat is secure and stable before using. Failure to comply with all product warnings may result in serious injury.
- **WARNING:** User **MUST** maintain even weight distribution while lowering and rising from device and while seated. Not doing so can result in tipping of the device and falls/serious injuries.

- **DO NOT** drop full body weight when lowering onto device as this could cause serious injury or damage to the device. Gently lower body weight when sitting onto the device.
- **NEVER** use the Toilet Seat Elevator on top of an existing toilet seat.
- **WARNING:** Sit on and leave seat from front, **NOT** from side. Make sure you have your balance before standing.
- **WARNING:** This product type is designed for use with the most common types of commodes. It will **NOT** fit all commode types. Be sure that the unit has been installed correctly, fits properly and is stable before use. If seat can rock from side to side, **DO NOT USE**.
- **WARNING: DO NOT** stand, kneel or climb on the Toilet Seat Elevator.
- **DO NOT** let children play on or operate the Toilet Seat Elevator.
- **WARNING:** Be sure that the user's weight is centered on the device during use. **NEVER** lean, reach or twist torso from side to side as injury may occur.
- **WARNING:** This is **NOT** a Locking Raised Toilet Seat. Be sure to center your weight on the Toilet Seat Elevator, as the seat may flip if you lean too far forward or to either side. Failure to comply may result in severe injury
- Check fit and tightness of the Toilet Seat Elevator to the bowl and all bolts and knobs at least weekly to ensure that the Toilet Seat Elevator is secure and safe to use. **NEVER** use the Toilet Seat Elevator if any looseness in the assembly or fit to the toilet bowl is noticed, as serious injury may result.
- It is recommended that metal bolts be used in the installation of the Toilet Seat Elevator as plastic bolts wear out quickly which could result in serious injury.

FAILURE TO USE COMMON SENSE AND HEED THE ABOVE WARNINGS FURTHER INCREASES RISK OF SERIOUS INJURY. USE AT YOUR OWN RISK AND WITH APPROPRIATE AND SERIOUS ATTENTION TO SAFE OPERATION. USE CAUTION. COMPASS HEALTH BRANDS CORP. ASSUMES NO RESPONSIBILITY FOR ANY DAMAGE OR INJURY CAUSED BY IMPROPER INSTALLATION, ASSEMBLY OR USE OF THIS PRODUCT.

Assembly / Operating Instructions

1

Remove Packaging and Check Parts


- ☐ Remove the Toilet Seat Elevator components from all packaging including tape, cardboard and plastic.
- ☐ To avoid risk of suffocation to children or animals, dispose of the plastic bags correctly.
- If you **DO NOT** understand the manual, contact your dealer for a complete explanation of the instructions, cautions and warnings. Otherwise injury may result.
- **DO NOT** use excessive force when installing any parts.
- If any parts are missing or damaged, **DO NOT** attempt to use.

TOOLS REQUIRED



Flathead Screwdriver

	Description	QTY
A	Toilet Seat Elevator	1
B	Bolts	2
C	Wing Nuts	2

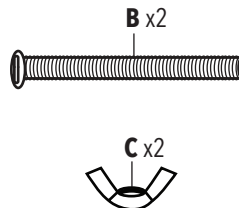

Specifications

Weight Capacity: 300 lbs


FGB30600 0000 Product DIMS: 3.5"h x 14"w x 19.25"d | Product Weight: 2.5 lbs

FGB30700 0000 Product DIMS: 3.5"h x 13.5"w x 17.5"d | Product Weight: 2.33 lbs




2 Installing Toilet Seat Elevator

- Unscrew existing toilet seat then line up the Toilet Seat Elevator (A) between that and the toilet bowl.
- Use provided Bolts (B) to go through the toilet seat, Toilet Seat Elevator (A), and the bowl.

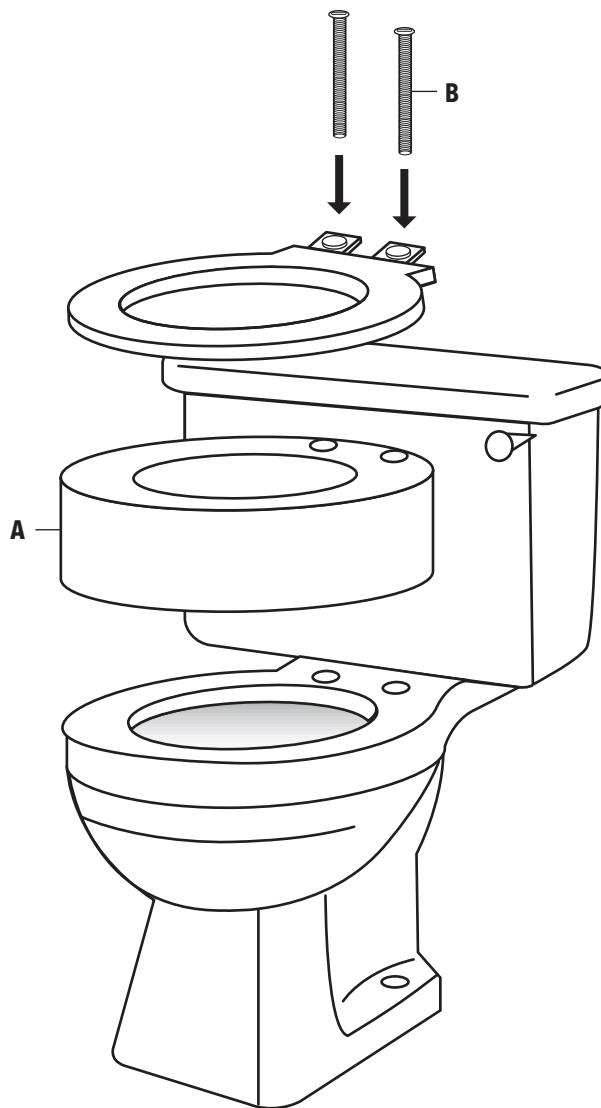

3 Screw On Wing Nuts To Secure Seat

- Attach both Wing Nuts (C) to the bottom of the Bolts to ensure the toilet seat and Toilet Seat Elevator are safely in place.

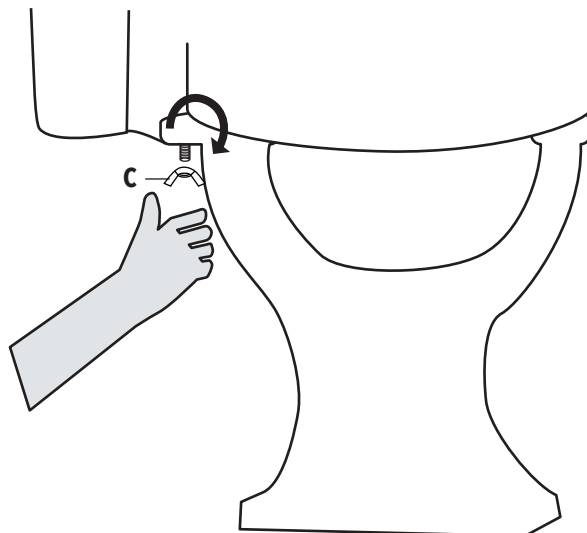


4 Toilet Seat Elevator Use

- Check before sitting: If the seat is unstable, check the bolts and secure.

⚠ WARNING: User **MUST** maintain even weight distribution while lowering and rising from device and while seated. Not doing so can result in tipping of the device and falls/serious injuries.

⚠ WARNING: If the device is unable to be securely installed (device shifts left to right or up/down lifting from bowl), **DO NOT USE**. Different commode styles may require alternate seating devices.

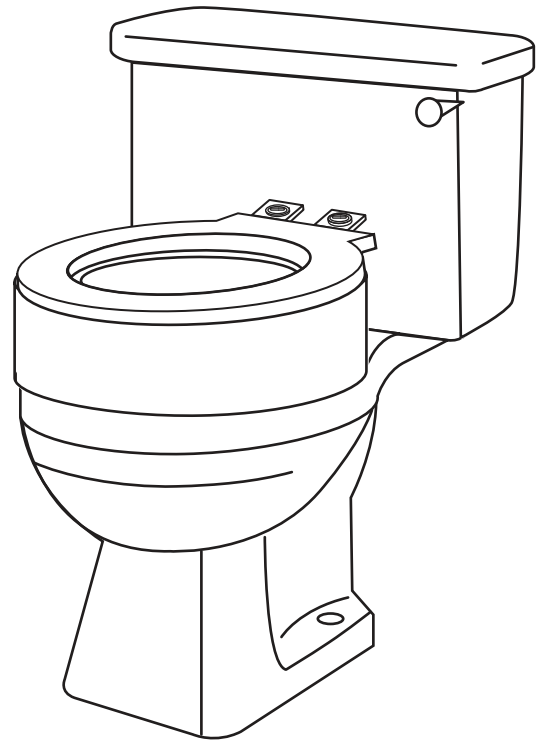

8 Cleaning and Maintenance

⚠ WARNING: The lack of care and maintenance may cause serious injury. Your device should be regularly inspected for signs of damage or wear and tear.

- Check fit and tightness of the Toilet Seat Elevator at least **WEEKLY** to ensure that the Toilet Seat Elevator is secure and safe to use. **NEVER** use the Toilet Seat Elevator if any looseness in the assembly is noticed, as serious injury may result.
- Use damp cloth and mild soap to clean the Toilet Seat Elevator. Ensure seat is completely dry before use.
- Ensure **ALL** crevices and seams are cleaned to avoid potential contaminants and prevent bacteria buildup.

CAREX[®]

Questions, problems or missing parts?

Before returning, contact Compass Health Brands customer service:

800-526-8051 | CAREX.COM

Retain this manual for future use.

COMPASSHEALTH

Manufactured for: Compass Health Brands Corp.
6753 Engle Rd., Middleburg Heights, OH 44130

www.compasshealthbrands.com • 800.376.7263

Made in China • 42-FGB30600 0000_FGB30700 0000_01 © 2025