


6" JUNIOR ROLLATOR (RLAJ6BG/BL)

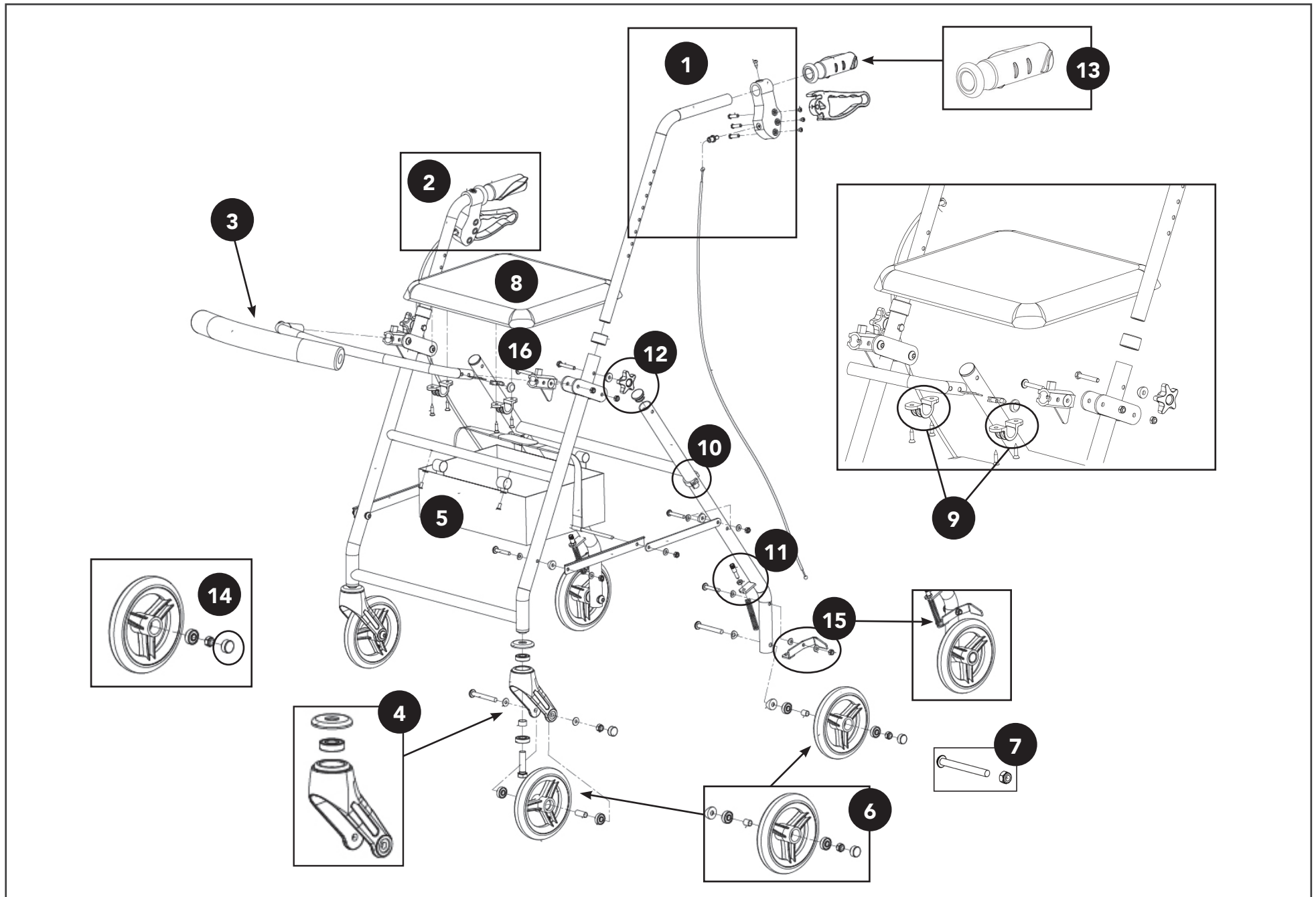

6" JUNIOR ROLLATOR (RLAJ6BG/BL)

Fig.	Description	MH Part No.
1	ProBasics 6" Alum. Jr Rollator Left Hand Brake Assembly (Includes brake spring & wheel contact)	X1RLAJ6MH01
2	ProBasics 6" Alum. Jr Rollator Right Hand Brake Assembly (Includes brake spring & wheel contact)	X1RLAJ6MH02
3	ProBasics 6" Alum. Jr Rollator Backrest (Hardware Not Included)	X1RLAJ6MH03
4	ProBasics 6" Alum. Jr Rollator Front Caster Fork Assembly	X1RLAJ6MH04
5	ProBasics 6" Alum. Jr Rollator Replacement Pouch	X1RLAJ6MH05
6	ProBasics 6" Alum. Jr Rollator 6" Caster w/Bearing (Front or Rear)	X1RLAJ6MH06
7	ProBasics 6" Alum. Jr Rollator Caster Axle Bolt w/Nut (Front or Rear)	X1RLAJ6MH07
8	ProBasics 6" Alum. Jr Rollator Replacement Seat	X1RLAJ6MH08
9	ProBasics 6" Alum. Jr Rollator Seat Clips and Screws	X1RLAJ6MH09
10	Probasics Brake Cable Ring	X1RLMH01
11	Probasics Rollator Brake Spring	X1RLMH02-4
12	Probasics Rollator Handle Height Adjustment Knob	X1RLMH03
13	Probasics Rollator Handle Grips, Pair	X1RLMH04
14	Probasics Rollator Caster Cap	X1RLMH05
15	Probasics Rollator Brake Wheel Contact	X1RLMH06-3
16	Probasics Rollator Backrest Fixator	X1RLBL07