

## Meridian Medical™ Aruba Therapeutic Foam Mattress Group 1


ARUBA-200 / UPC: 0 92237 61623 9 / 80"L x 36"W x 6"H / Weight Cap. 350 lbs.

ARUBA-300 / UPC: 0 92237 61624 6 / 84"L x 36"W x 6"H / Weight Cap. 350 lbs.



**DEALER:** This manual **MUST** be given to the user of the product.

**USER:** **BEFORE** using this product, read this manual and **SAVE** for future reference.

 **BEFORE TRANSFERRING OWNERSHIP OF THE PRODUCT TO A DIFFERENT USER ALWAYS HAVE A DEALER INSPECT THE PRODUCT FOR ANY DAMAGE. IF ANY DAMAGE IS FOUND, DO NOT USE THE PRODUCT. PRODUCT MUST ALWAYS BE ACCOMPANIED WITH THESE INSTRUCTIONS AND WARNINGS. FAILURE TO HEED INSTRUCTIONS AND WARNINGS INCREASES RISK OF INJURY. DO NOT ATTEMPT TO USE THE MATTRESS WITHOUT READING ALL INSTRUCTIONS AND WARNINGS CAREFULLY. FAILURE TO COMPLY MAY RESULT IN SERIOUS INJURY.**

### Indications for Use

The Meridian Medical Group 1 Therapeutic Foam Mattress is intended to be used with an appropriately sized bed frame as a part of an overall pressure injury prevention program. Any other use is prohibited.

This pressure redistribution mattress is designed to positively contribute to the outcome of a pressure injury prevention care plan when the product is in normal use as defined by Compass Health Brands: the support surface is covered with a cotton, cotton combination, or linen bed sheet, and any one of these is the only item between the support surface and the user. This product should **ALWAYS** be used in conjunction with informed nursing judgements when assessing risk from pressure injury development.


### Contraindications

There are no known contraindications for use of this product.

### Warnings

- **DO NOT** expose to direct sunlight.
- **DO NOT** soak in water.
- **DO NOT** leave any heavy and hard objects on the mattress.
- **DO NOT** use if damaged.
- **KEEP** away from corrosive chemicals and flammable sources.
- **No Smoking in Bed:** Smoking in bed can be dangerous. To avoid the risk of fire, smoking in bed should **NEVER** be allowed.
- Because of potential risk of infectious exposure, cleaning with the patient on the bed is **NOT** recommended. Any item visibly soiled with any bodily fluids should be properly cleaned or removed and all nearby equipment should be inspected for contamination. Staff members should wear appropriate protective clothing when cleaning soiled bedding. Follow standard institutional infection control procedures.
- **DO NOT** open the zippered mattress top cover during cleaning, infection control procedures, or during use.
- **DO NOT** launder or autoclave mattress covers.
- With use, increased risk of patient entrapment is possible due to mattress compression. At regular intervals monitor gaps between the mattress, bed, and/or rail. Mattresses **MUST** fit bed frame and bed rails and if gaps are present, the mattress should be replaced.
- Visit the FDA website at <http://www.fda.gov> to learn about the risks of entrapment. Refer to the owners' manuals for beds and rails for additional safety information.

### Instructions for Use

 **WARNING - DO NOT** use sharp tools to open the mattress.

1. Remove the mattress from its packaging and unroll the mattress. Completely close the zippered cover.
2. Allow the mattress to decompress fully before use (this may take up to 8 hours).
3. Place non-slip vinyl side face down on the bed surface. Printed side of mattress should be faceup.
4. For best results rotate the mattress (head area to foot area) every 6 months. Ensure the printed side is **ALWAYS** faceup.

## Meridian Medical™ Aruba Therapeutic Foam Mattress Group 1

ARUBA-200 / UPC: 0 92237 61623 9 / 80"L x 36"W x 6"H / Weight Cap. 350 lbs.

ARUBA-300 / UPC: 0 92237 61624 6 / 84"L x 36"W x 6"H / Weight Cap. 350 lbs.



---

### Cleaning & Maintenance

The mattress **MUST** be cleaned regularly and/or between use with different patients.

- Remove any bedding from the mattress.
- Because of potential risk of infectious exposure, cleaning with the patient on the bed is **NOT** recommended. Any item visibly soiled with any bodily fluids should be properly cleaned or removed and all nearby equipment should be inspected for contamination. Staff members should wear appropriate protective clothing when cleaning soiled bedding. Follow standard institutional infection control procedures.
- Wipe down both the top and bottom of the mattress covers using a damp cloth pre-soaked in a standard cleaning solution of warm soapy water containing a mild detergent.
- Intermediate disinfectants are acceptable for use on the cover materials. Top covers consisting of typically polyester and/or polyurethane are recommended to be disinfected with Quaternary. Diluted chlorine bleach solution (1:10) is also acceptable. Phenolics are **NOT** recommended for poly fabric materials.
- Bottom covers consisting of typically vinyl materials are recommended to be disinfected with Phenolics. Diluted chlorine bleach solution (1:10) is also acceptable. Quaternary and/or isopropyl is **NOT** recommended.
- All components of the mattress should be air-dried thoroughly outside of proximity to dusty areas before re-using.
- The cover should be checked when being cleaned and replaced, as necessary, due to excessive wear, rips or other defects that may impact the performance and safety of the mattress.
- The lack of care and maintenance may cause serious injury. Your device should be regularly inspected for signs of damage or wear and tear. Immediately replace if damage exists.
- Check mattresses surface for tears or cracking; **DO NOT** use if tears or cracking are present. If any damages exist, please contact your local provider immediately.
- **Disposal:** At the end of useful life, dispose of waste according to local requirements.

### Limited Warranty

**This warranty is solely for the benefit of the original consumer purchaser.**

This device was built to exacting standards and was carefully inspected prior to shipment. Your Meridian Medical™ brand product is guaranteed to be free of defects in materials and workmanship for a two-year warranty period on the foam core from the date of purchase and a one-year warranty on the mattress cover, which is subject to normal wear and may require periodic replacement. If within such warranty period, the product proves defective, the product will be serviced or replaced, at the option of Compass Health Brands. This warranty does not cover product failures due to normal wear and tear or device misuse.

In the event of a defect covered by this warranty, we will, at our option, service or replace the device. This warranty provides only for replacement of defective parts and does not cover shipping or labor charges. Compass Health Brands' sole obligation and your exclusive remedy under this warranty is limited to such service and/or replacement.

This warranty shall not apply to serial numbered products if the serial number has been removed or defaced or to defects or damages caused by unauthorized parts, attachments or repairs, negligence, accident, circumstances beyond Compass Health Brands' control, or improper operation, maintenance or storage.

If you have a question about your Compass Health Brands product or this warranty, please contact an authorized Compass Health Brands Dealer.

Manufactured for:  
Compass Health Brands  
6753 Engle Road, Middleburg Heights, OH 44130  
Tel: (800) 376-7263  
[www.compasshealthbrands.com](http://www.compasshealthbrands.com)

Made in USA