

Ascent Backrest Assembly Instructions

Congratulations on purchasing your Ascent Rollator! Upon unboxing your rollator, you will see a rollator frame with a bag attached and a bag with a backrest and hardware. This guide will take you step by step on attach the backrest in a few quick seconds while also setting the height of the backrest for maximum comfort.

Please refer to your included user guide for any additional questions regarding the Ascent rollator.

Easy Assembly

All parts conveniently packed in a single box.
No additional tools required.

- 1** Locate the backrest and hardware and remove from box. The contents will look like the images below:

This will include:
1 - Backrest
2 - Locking Dials



- 2** Remove locking dials (if attached) from backrest bracket. Separate the plastic ends of the backrest bracket and slide the bracket onto the rollator arms shown below:



- 3** Once the backrest is on the rollator arms, the backrest can be moved up or down to set the desired height. Once the desired height is determined, locate the locking dials. Using the dials, lock them into place turning them clockwise until finger tight.



- 4** Final Product with Backrest attached:

