Carex Health Brands

Part No.
L'article No **B313-00**
Nº de pieza

Maximum user weight:
Poids maximum de l'utilisateur:
Peso máximo del usuario:

500 lbs
220 kg

B313 Safe Lock Raised Toilet Seat

ENGLISH

Can be fitted to a wide variety of toilet sizes including elongated and round-front bowls.

1. Fit the brackets (see diagrams). Lift existing toilet seat and place the raised seat loose on to the toilet bowl.
2. The front bracket is used to position the seat from front to back on the bowl. With the seat positioned centrally this adjustment should be carried out before the side brackets are tightened up, not afterwards!
3. Screw the side brackets in clockwise, keeping the seat central on the bowl whilst keeping the front bracket pushed up to the front. Now tighten the side brackets, not the front.
4. Ensure the seat is a tight fit to the bowl. Test for stability before use.
5. The product is designed for use in a static mode only. As with all devices designed to assist the elderly or disabled, care should be taken to avoid sudden or undue movement during use.
6. To clean use water and detergent with disinfectant. Never use abrasive cleaners or clean whilst dry.

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Carex Health Brands

B313 Safe Lock Raised Toilet Seat

Fitting The Brackets & Front Bracket

PARTS


1 Front bracket


2 brackets

1 "L" Key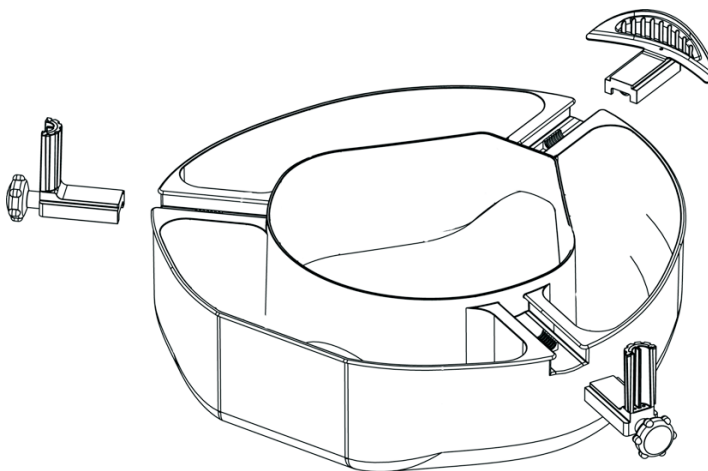

Fit the brackets by screwing the side knobs clockwise.
If needed fit the Front Bracket using the "L" key provided.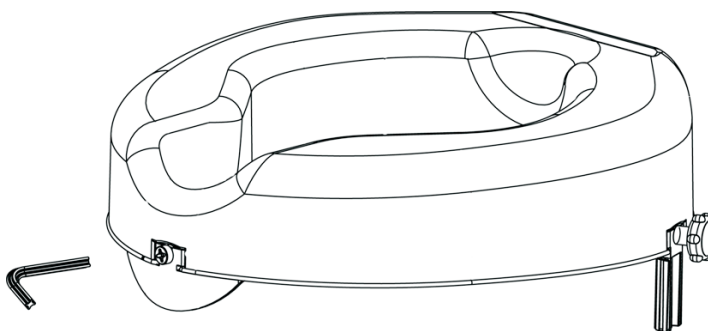

Only use the "L" key provided to make adjustments
to the Front Bracket.

Carex Health Brands

PO Box 2526, Sioux Falls, SD 57101-2526
Phone: 800-328-2935 • Fax: 888-616-4297
www.carex.com
40-B313-00_00

DO NOT RETURN THIS PRODUCT TO STORE

Any questions or concerns about any aspect of this product need
to be directed to our customer service at toll free number: **1-800-526-8051**.
We are available Monday – Friday, 8AM – 5PM CST, to address your needs.