
Bariatric Shower Chair

Silla de ducha bariátrica/
Chaise de douche bariatrique

BARIATRIC SHOWER CHAIR DIAGRAM

Silla de ducha bariátrica/Chaise de douche bariatrique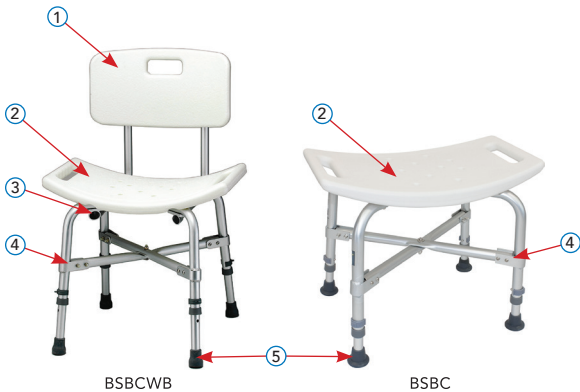

WEIGHT CAPACITY:

550 lb/249 kg (BSBCWB, BSBC)

Capacidad de peso / Capacité de poids

	English	Español	Français
1	Back support*	Soporte para la espalda*	Dossier*
2	Seat	Asiento	Siège
3	Channel guide	Guía del canal	Tube de guidage
4	Height adjustable leg with u-clip	Pata de altura regulable con clip en u	Pattes à hauteur réglable avec attaches en u
5	Leg tip	Puntas de la pata	Embouts

*The back support is included with item #BSBCWB.

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Congratulations on purchasing a quality ProBasics® Bariatric Shower Chair. Our shower chairs offer stability, comfort and easy assembly. Please be sure to read these instructions and warnings thoroughly and follow the instructions of your healthcare provider to maximize the shower chair's benefits and your safety.

DO NOT ATTEMPT TO ADJUST OR OPERATE THIS SHOWER CHAIR WITHOUT READING ALL INSTRUCTIONS CAREFULLY.

ASSEMBLY INSTRUCTIONS

Remove the shower chair components from all packaging including tape, cardboard and plastic. Check all parts for damage before use. If any parts are missing or damaged, **DO NOT** attempt to use the shower chair. If you do not understand the manual, contact your dealer for a complete explanation of the instructions, cautions and warnings. Otherwise injury may result.

NOTE: DO NOT use excessive force when installing any parts.

SETUP INSTRUCTIONS

1. Remove seat frame, legs and back support (BSBCWB only) from carton.
2. Slide back support through channel guides until back support tube buttons protrude fully through channel guide holes and lock into place. An audible click will be heard.
3. Connect the height adjustable legs to the leg tubes under the seat, ensuring all legs are adjusted to the same setting.
4. Adjust seat height as directed in the "Height Adjustment" section.

HEIGHT ADJUSTMENT

1. Remove u-clip pin from leg frame. Insert height adjustable leg attachment into leg frame to desired position, lining up holes.
2. Secure u-clip pin through height setting until the clip snaps firmly into place around the leg. An audible click will be heard.
3. Repeat for other three leg attachments.
4. To ensure legs are locked into place, push down on seat or pull gently on each leg.

CAUTION: Ensure that all four shower chair legs are adjusted to the same height, and that all height adjustment buttons protrude fully through adjustment holes before use. **DO NOT** sit on shower chair if legs are not evenly adjusted, locked and stable.

MAINTENANCE

1. Use a mild soap and water solution, rinse with clean water, then dry with a soft cloth. Ensure that the shower chair is well rinsed before using.
2. Never use organic solvents, abrasive cleaners or scouring pads on any part of shower chair.
3. Check fit and tightness of all screws, nuts, bolts and pins **WEEKLY** to ensure that the shower chair is safe to use.

WARNINGS

DO NOT ATTEMPT TO ADJUST OR OPERATE THE SHOWER CHAIR WITHOUT READING ALL INSTRUCTIONS CAREFULLY.

- Maximum weight capacity: 550 lb/249 kg (BSBCWB, BSBC).
- **DO NOT** use this device if it is not securely installed.
- This product should not be used without instruction from a healthcare professional.
- The seat and legs must be securely attached to the frame prior to use. Failure to do so may result in injury or damage to the shower chair.
- Ensure the height adjusting snap buttons fully protrude through the same respective hole of each leg extension. This will ensure the leg extensions are locked in position and an even height is achieved.
- All four leg extensions with rubber tips must touch the floor at all times.
- Ensure **ALL** screws, nuts and/or bolts are tightened and threading is intact on a weekly basis.
- Check all the rubber tips on the leg extensions for rips, wear or if they are missing. Immediately replace any or all if any of these imperfections exist.
- Weight should be evenly distributed when using the shower chair. **DO NOT** lean or reach for objects while seated on the shower chair.
- **NEVER** stand on the shower chair seat.
- It is recommended that the user ease down onto the seat. **DO NOT** sit on the seat with direct force.
- Always use caution when surfaces are wet.
- Inspect all parts for shipping damage before assembly. If there is shipping damage — **DO NOT USE**. Contact dealer/carrier for further instructions.
- Users with limited physical capabilities should be supervised or assisted in shower or tub areas even when using support rails.

- Accessories designed by other manufacturers have not been tested by Compass Health Brands and are not recommended for use with this product.
- Consult a healthcare professional for advice and instructions specific to your needs before use.
- Compass Health Brands assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.

USE CAUTION. FAILURE TO USE COMMON SENSE AND HEED THE ABOVE WARNINGS FURTHER INCREASES RISK OF SERIOUS INJURY. USE AT YOUR OWN RISK AND WITH APPROPRIATE AND SERIOUS ATTENTION TO SAFE OPERATION.

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NOTES / NOTAS / REMARQUES

LIMITED LIFETIME WARRANTY

This warranty is solely for the benefit of the original consumer purchaser.

This device was built to exacting standards and was carefully inspected prior to shipment. Your ProBasics® brand product is warranted to be free of defects in materials and workmanship, has a Limited Lifetime Warranty on the frame and a one year warranty from date of purchase on all other parts, which are subject to normal wear and need periodic replacement. If within such warranty period, the product proves defective, the product will be repaired or replaced, at the option of Compass Health Brands. This warranty does not cover product failures due to normal wear and tear or device misuse.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty provides only for replacement of defective parts and does not cover shipping or labor charges. Compass Health Brands' sole obligation and your exclusive remedy under this warranty is limited to such repair and/or replacement.

This warranty shall not apply to serial numbered products if the serial number has been removed or defaced or to defects or damages caused by unauthorized parts, attachments or repairs, negligence, accident, circumstances beyond Compass Health Brands' control, or improper operation, maintenance or storage.

If you have a question about your Compass Health Brands product or this warranty, please contact an authorized Compass Health Brands Dealer.

COMPASSHEALTH

6753 Engle Road
Middleburg Heights, Ohio 44130
Tel.: (800) 376-7263
www.compasshealthbrands.com