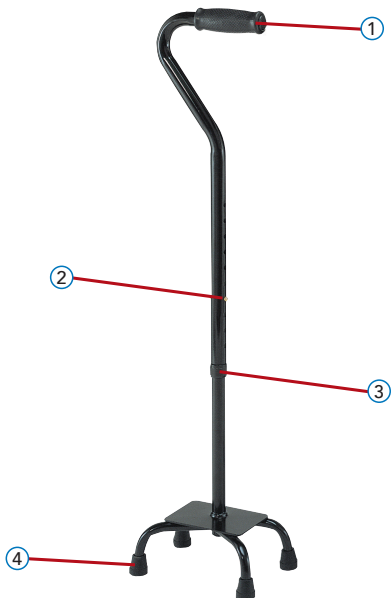
# Quad Cane

# QUAD CANE DIAGRAM

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# OPERATING INSTRUCTIONS

To ensure the safe operation of ProBasics® Canes, these instructions must be followed.

## PROPER CANE HEIGHT

1. Stand as upright as possible on a firm, flat surface and let your arm hang down naturally by your side.
2. Adjust cane so top portion of handle touches inside of wrist where it meets palm.

**NOTE:** For a more comfortable fit, ensure shoe height is the same as shoe style worn most frequently when using cane.

## HEIGHT ADJUSTMENT

1. Loosen the locking ring by twisting clockwise.
2. Adjust the cane height by depressing the brass push button and sliding the lower section upward or downward to desired height.

**NOTE:** An audible click will be heard when push button securely protrudes through the adjustment hole.

3. Repeat steps 1-2 until cane is at proper height.
4. Tighten locking ring by turning counter-clockwise.

## ADJUSTING FOR LEFT/RIGHT HAND USE

1. Loosen the locking ring by twisting clockwise.
2. Depress the push button and turn cane handle 180 degrees until the push button locks into same height adjustment hole on other side of cane.
3. Secure locking ring by twisting counter-clockwise until tight.

## MAINTENANCE

1. Ensure the snap button releases and re-engages properly.
2. Inspect cane tips for wear and damage.
3. Replace any broken, damaged or worn items immediately.

## WARNINGS

### **DO NOT ATTEMPT TO ADJUST OR OPERATE CANE WITHOUT READING ALL INSTRUCTIONS CAREFULLY.**

- The cane is intended to be used solely as a walking aid and all other use can cause risk of injury.
- This product has a maximum user weight limitation of 300 lb/136 kg (CNQSB, CNQLB, CNQSBS, CNQLBS), 500 lb/227 kg (CNQHSBB, CNQHLBB). Canes are **NOT** designed to support the total weight of individuals.
- Before use, ensure cane tip is properly seated and straight. Check rubber tip for rips, tears, cracks or wear. If any of these conditions exist, replace rubber tip immediately.
- Before use, the height adjustment push button **MUST** be fully engaged and protruding through height adjustment hole on cane to ensure a positive lock. **ALWAYS** test to see that cane handle is properly and securely locked in place or a fall could result, causing bodily injury and/or damage to the cane.
- Make sure locking ring is securely tightened at all times.
- This cane is **NOT** intended for use on stairs.
- Avoid wet floors, ice, soft ground and similar hazards.
- When using a Quad cane, the cane should be placed on a level walking surface with all four legs in contact with the surface. Use of cane in a fashion other than this can lead to eventual cane damage and/or personal injury.
- For stability, ensure that shorter legs of the base are closest to foot - otherwise, injury or damage may occur. It may be necessary to adjust cane for right-hand or left-hand use.
- If components are damaged or missing, contact your dealer immediately. **DO NOT** use substitute parts.
- Failure to follow these instructions may result in serious injury.
- Compass Health Brands assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.

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# LIMITED LIFETIME WARRANTY

This ProBasics® product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to precise standards and carefully inspected prior to shipment. This Limited Lifetime Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to our customers for years of dependable service.

Your ProBasics® brand product carries a limited lifetime warranty on the frame and 90 days on all other non-durable components including but not limited to rubber accessories, and grips, which are subject to normal wear and need periodic replacement. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty provides only for replacement of defective parts and does not cover shipping or labor charges. Compass Health Brands' sole obligation and your exclusive remedy under this warranty is limited to such repair and/or replacement.

This warranty does not cover product failures due to normal wear and tear or device misuse. This warranty shall not apply to serial numbered products if the serial number has been removed or defaced, or to defects or damages caused by unauthorized parts, attachments or repairs, negligence, accident, circumstances beyond Compass Health Brands' control, or improper operation, maintenance or storage.

If you have a question about your Compass Health Brands product or this warranty, please contact an authorized Compass Health Brands Dealer.

**COMPASSHEALTH**

6753 Engle Road  
Middleburg Heights, Ohio 44130  
Tel.: (800) 376-7263

[www.compasshealthbrands.com](http://www.compasshealthbrands.com)