



Classics



Classics


rollator

rollator

- Height adjustable with locking hand brakes
- Padded 12" seat and backrest
- Convenient storage bag
- Large 8" wheels for increased maneuverability
- Sturdy steel frame is easy to fold for simple travel and storage



6.15"

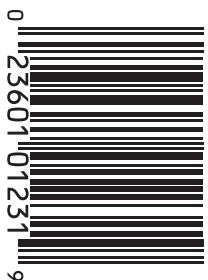

A23177 0000

### COMPASSHEALTH

Manufactured for / Fabricado por:  
 Compass Health Brands Corp.  
 Middleburg Heights, OH 44130  
[www.compasshealthbrands.com](http://www.compasshealthbrands.com)  
 Phone / Teléfono 800.376.7263  
 Made in China / Hecho en China  
 41-FGA23177 0000\_03 © 2021

Height adjustable with  
locking hand brakes  
Weight capacity: 300 lbs.

7"

### COMPASSHEALTH

CONTACT: 781.871.2710  
 FILE NAME: FGA23177 0000\_SteelRollator\_03  
 FACTORY: Valentine  
 DATE: July 26, 2021


### PRINTING INFORMATION

MATERIAL: 120 lb Cover Gloss  
 FINISH: -  
 SIZE INCHES: 6.15" h x 7" w  
 SIZE MM: -

### COLOR LEGEND: 4 Color

Cyan  
 Magenta  
 Yellow  
 Black

## Adjusting the handles and seat height

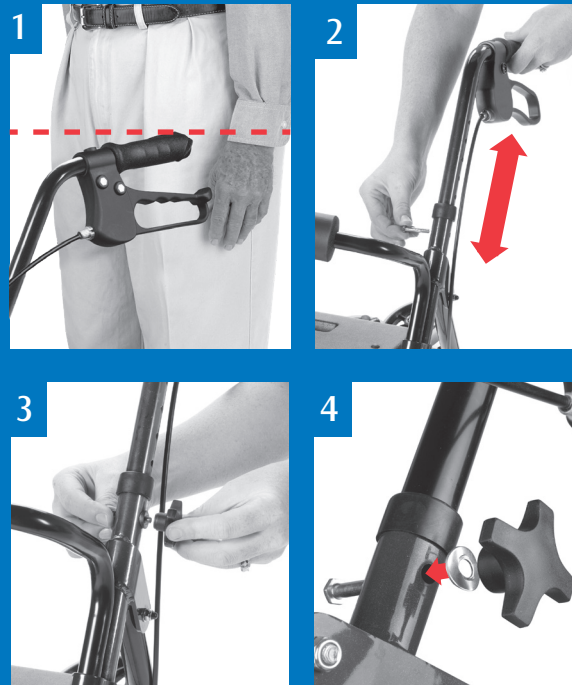

Carex® offers several styles of rollators and accessories for easy mobility, maneuvering, and convenience.

### HOW DO I ADJUST MY ROLLER WALKER?

1. Stand with your shoulders relaxed and your arms hanging loosely at your sides.
2. The handlebar height should be at the crease of your wrist when your arms are relaxed at your sides.
3. Loosen the knobs on the outside of the handlebars, remove the bolts, then move the handlebars to the desired height.
4. Re-install the bolts and screw the knobs to the bolt. Ensure the bolt head is properly aligned with the hole before tightening fully.
5. When sitting on the seat the balls of your feet should touch the ground.

### HOW DO I USE MY ROLLER WALKER?

1. Push the walker forward, allowing it to roll forward with control as you step forward.
2. To slow the walker down, squeeze or pull up on the handbrake.
3. Before using the seat, activate the parking brake.
4. To release the brake, carefully squeeze the brake handles again, being careful not to pinch your fingers.

6.15"

7"