

# CAREX<sup>®</sup>

ITEM# B31000 0000

## Raised Toilet Seat



 **Please read all instructions before assembly or use.**

Questions, problems or missing parts?  
Before returning, contact Compass Health Brands customer service:

**800-526-8051 | CAREX.COM**

# WARNINGS

## INDICATIONS FOR USE:

A raised toilet seat is intended to increase the height of the seating surface and can be used by persons with decreased strength or mobility. This device is intended for a single user **ONLY**.


## CONTRAINDICATIONS:

For safety reasons, people with balance disorders should **NOT** remain without supervision when using this device. User **MUST** be able to maintain even weight distribution while lowering and rising from device and while seated.

This device **MUST NOT** be used by individuals whose feet **DO NOT** touch the floor when seated, as appropriate balance on the device will not be possible.

**SAVE THESE INSTRUCTIONS. This sheet MUST accompany the device with each use.**

## INSTRUCTIONS FOR USE:

 **WARNING:** User **MUST** maintain even weight distribution while lowering and rising from device and while seated. Not doing so can result in tipping of the device and falls/serious injuries.

 **WARNING:** If the device is unable to be securely installed (device shifts left to right or up/down lifting from bowl), **DO NOT USE**. Different commode styles may require alternate seating devices.

## WARNINGS:

- **DO NOT ATTEMPT TO ADJUST OR OPERATE THE RAISED TOILET SEAT WITHOUT READING ALL INSTRUCTIONS CAREFULLY.**
- Inspect all parts for shipping damages before assembly. If there is shipping damage – **DO NOT USE**. Contact your dealer/carrier for further instructions.
- If you **DO NOT** understand these instructions, contact a healthcare dealer or technical professional before attempting to install or use the device.
- Users that are unable to understand or follow these instructions and warnings or have limited physical capabilities should be supervised or assisted while using the raised toilet seat.
- This raised toilet seat has a weight limit of 300 lbs. (136 kg). **ALWAYS** observe the weight limit on the labeling of your raised toilet seat.
- Use the toilet lift only for the purpose for which it was designed. Compass Health Brands is **NOT** responsible in any way for any misuse of the raised toilet seat, or for injuries or damage caused as a result of it.
- Check that all labels are present and legible. Replace if necessary.
- **DO NOT** use this device if your feet cannot make contact with the floor when in a seated position, as this will change the weight distribution and stability of the device, which can result in serious injury.
- Check fit and tightness of the raised toilet seat to the bowl and all bolts and knobs at least weekly to ensure that the raised toilet seat is secure and safe to use.
- It is recommended that a toilet safety frame or permanently installed wall mount grab bar is used in addition to the raised toilet seat. Discuss your needs with your health care provider.
- **DO NOT** use the raised toilet seat if wobbly, unstable, or not level. **ALWAYS** be sure that the seat is secure and stable before using. Failure to comply with all product warnings may result in serious injury.
- This product is **NOT** suitable for use as a grab rail and/or stability aid while transferring to or from the toilet seat.
- **NEVER** use the raised toilet seat on top of an existing toilet seat.
- Sit on and leave seat from front, not from side. Make sure you have your balance before releasing your grasp from the raised toilet seat to stand.
- This product type is designed for use with the most common types of commodes. It will **NOT** fit all commode types. Be sure that the unit has been installed correctly, fits properly and is stable before use. If seat can rock from side to side, **DO NOT USE**.
- **DO NOT** stand, kneel or climb on the raised toilet seat.
- **DO NOT** let children play on or operate the raised toilet seat.
- Be sure that the user's weight is centered on the device during use. **NEVER** lean, reach or twist torso from side to side as injury may occur.

**FAILURE TO USE COMMON SENSE AND HEED THE ABOVE WARNINGS FURTHER INCREASES RISK OF SERIOUS INJURY. USE AT YOUR OWN RISK AND WITH APPROPRIATE AND SERIOUS ATTENTION TO SAFE OPERATION. USE CAUTION.**

## 1 Remove Packaging and Check Parts

- Remove the raised toilet seat from all packaging including tape, cardboard and plastic.
- To avoid risk of suffocation to children or animals, dispose of the plastic bags correctly.
- If you **DO NOT** understand the manual, contact your dealer for a complete explanation of the instructions, cautions and warnings. Otherwise injury may result.

	Description	QTY
A	Raised Toilet Seat	1



### Specifications

Dimensions: 16" (D) x 5.5" (H) x 15" (W) | Product Weight: 4.5 lbs. | Weight Capacity: 300 lbs.

## 2 Installation and Use

- Lift toilet seat and ensure rubber pads are making firm contact with rim. If device is not stable once installed, **DO NOT** use.
- When using the device maintain even weight across toilet seat. **DO NOT** lean or reach while using this device, as this may result in the seat tipping.

## 3 Cleaning and Maintenance

**⚠ WARNING: The lack of care and maintenance may cause serious injury. Your device should be regularly inspected for signs of damage or wear and tear.**

- Check fit and tightness of the raised toilet seat to the bowl and all bolts and knobs at least **WEEKLY** to ensure that the raised toilet seat is secure and safe to use. **NEVER** use the raised toilet seat if any looseness in the assembly or fit to the toilet bowl is noticed, as serious injury may result.
- Use damp cloth and mild soap to clean raised toilet seat. Ensure seat is completely dry before use.

# CAREX<sup>®</sup>

Questions, problems or missing parts?

Before returning, contact Compass Health Brands customer service:

**800-526-8051 | CAREX.COM**

**Retain this manual for future use.**

**COMPASSHEALTH**

Manufactured for:

Compass Health Brands Corp.  
Middleburg Heights, OH 44130  
www.compasshealthbrands.com  
Phone 800.376.7263

Made in China

42-FGB31000 0000\_00 © 2024