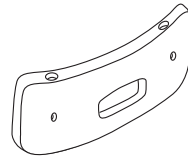
B753

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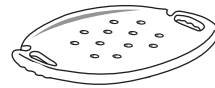
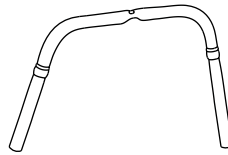
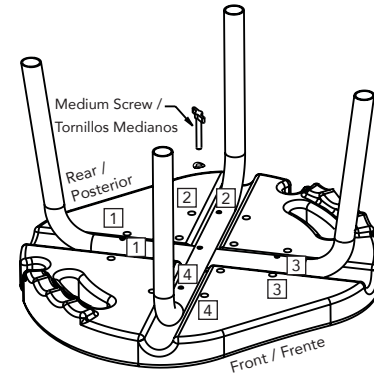
Figure 3. Securing U bars to seat
Figura 3. Fijación de barras U para asiento

1 Place seat face-down on a table or counter with the FRONT edge closest to you (BACK edge has the shower sprayer notches, FRONT edge does not). Place U bar labeled "1" in the groove in the seat which extends from BACK LEFT corner of seat to RIGHT FRONT corner of seat, as shown in Figure 1., above. Place the U bar labeled "2" in the other groove, on top of U bar "1." **NOTE:** when placed correctly, the 2 U bars will align at the center as shown in Figure 2., above and the "1", "2", "3" and "4" labels will all be aligned.

⚠ **WARNING:** Failure to assemble this bath seat correctly can result in falls / injuries.

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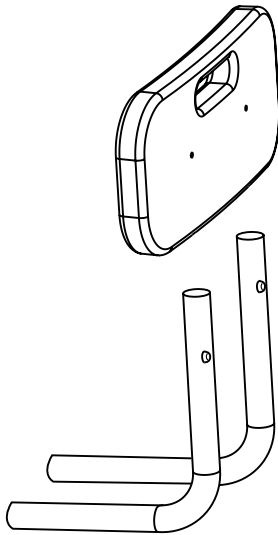
2 Find the 1 medium sized screw -
⚠ **WARNING** - Medium screw **MUST** be used for the next step or damage to the seat will occur. Place medium screw with washer into center hole through both U bars and into the screw hole in the seat. The contour of the washers should match the curve of the bar. **DO NOT** fully tighten yet.

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

- 3** Attach backrest to both L bars. Press in the pop pins and line up with backrest holes. Assure pins are completely through the holes.

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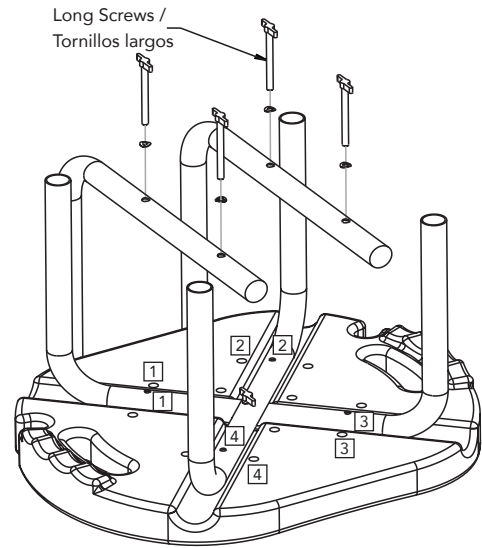
- 4** Place L bar over U bar holes so holes line up. Insert screws and washers on L bar. **DO NOT** tighten yet. Repeat with other L Bar. Fully tighten the 4 long screws. Tighten the middle medium screw. Periodically check to assure that all screws remain tight.

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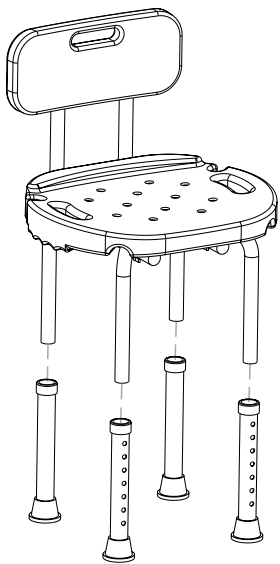- 5** Slip leg extensions on, push copper pin and slide on, line up pin and hole so pin pops through leg extension hole completely. Repeat for all 4 legs. Adjust height so all legs are the same height.

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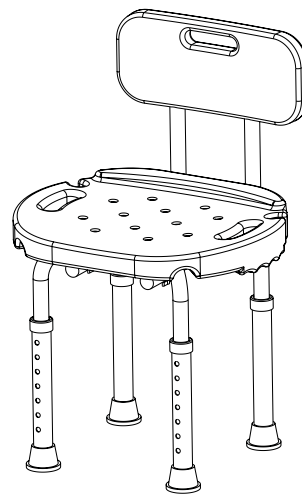- 6** Check for stability. Assure all pins are completely through the holes and legs are all the same height. Place Bath and Shower Seat in shower or bath on a level surface. Periodically check to assure that all screws remain tight.

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See additional warnings on enclosed warnings guide.

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