


K4 WHEELCHAIR

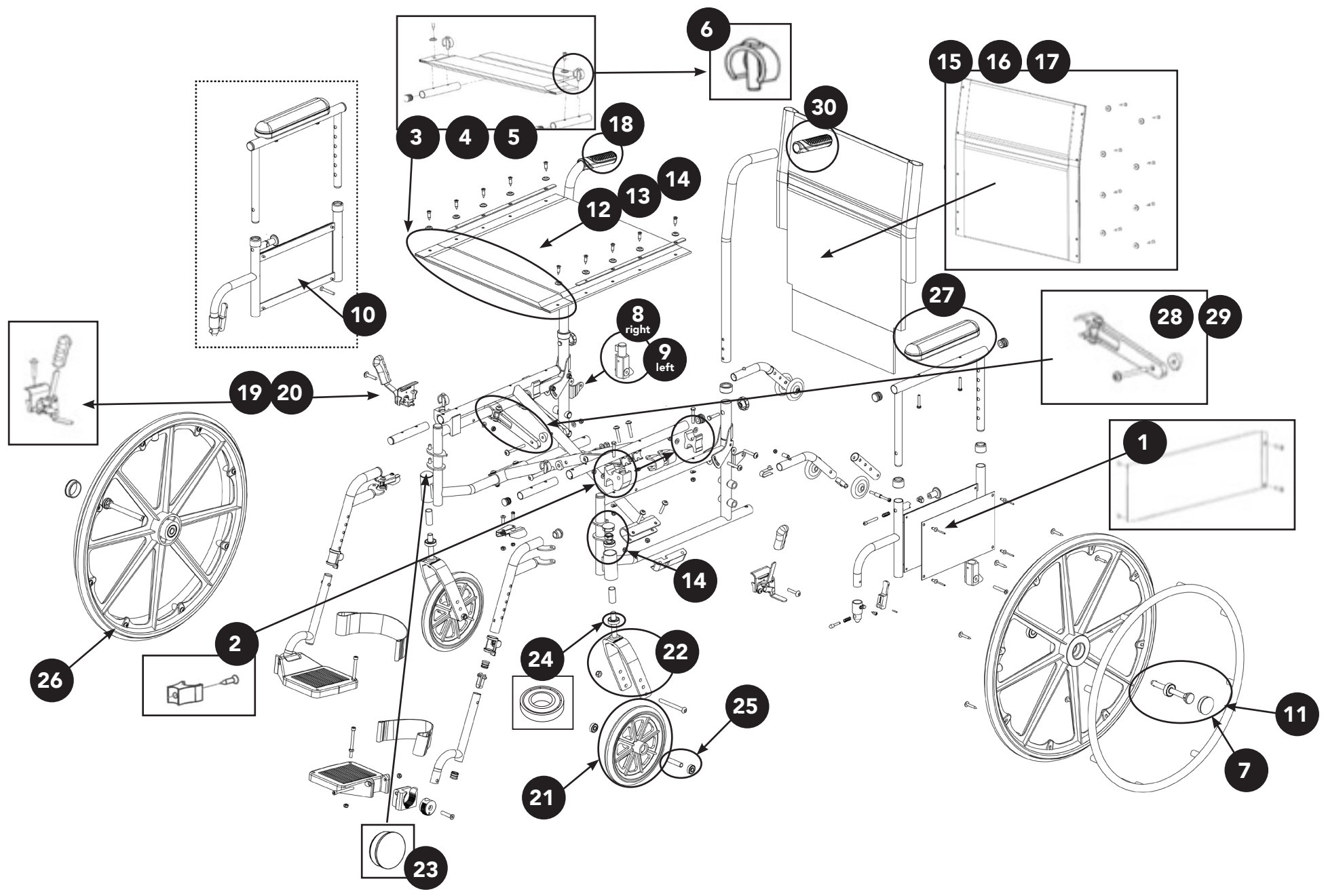

K4 WHEELCHAIR

Fig.	Description	Part No.
1	Probasics K4 Wheelchair Skirt Guard, Pair w/Hardware	X1WC4AS01
2	Probasics K4 Wheelchair Seat Rail Guide, Pair	X1WC4AS02
3	Probasics K4 Wheelchair 16" Seat Extender Attachment w/Seat Tube	X1WC4AS03
4	Probasics K4 Wheelchair 18" Seat Extender Attachment w/Seat Tube	X1WC4AS04
5	Probasics K4 Wheelchair 20" Seat Extender Attachment w/Seat Tube	X1WC4AS05
6	Probasics K4 Wheelchair Clip for Seat Extender Attachment	X1WC4AS06
7	Probasics K4 Wheelchair Quick Release Axle	X1WC4AS07
8	Probasics K4 Wheelchair Right Flip-Back Armrest Guides	X1WC4AS08
9	Probasics K4 Wheelchair Left Flip-Back Armrest Guides	X1WC4AS09
10	Probasics K4 Wheelchair Armrest Desk Assembly, Pair	X1WCAS18
11	Probasics K4 Wheelchair Rear Wheel Axle Bolt w/Nut	X1WC4AS10
12	Probasics K4 Wheelchair 16" Seat Upholstery w/Hardware	X1WC4AS11
13	Probasics K4 Wheelchair 18" Seat Upholstery w/Hardware	X1WC4AS12
14	Probasics K4 Wheelchair 20" Seat Upholstery w/Hardware	X1WC4AS13
15	Probasics K4 Wheelchair 16" Back Upholstery w/Hardware	X1WC4AS14
16	Probasics K4 Wheelchair 18" Back Upholstery w/Hardware	X1WC4AS15
17	Probasics K4 Wheelchair 20" Back Upholstery w/Hardware	X1WC4AS16
18	Probasics K4 Hand Grips, Pair	X1WC4AS17
K3/K4 REPLACEMENT PARTS		
19	Probasics K3/K4 Wheelchair Right Side Wheel Lock	X1WCAS03
20	Probasics K3/K4 Wheelchair Left Side Wheel Lock	X1WCAS04
21	Probasics K3/K4 Wheelchair Front Caster w/Bearings	X1WCAS05
22	Probasics K3/K4 Wheelchair Front Fork Assembly	X1WCAS06
23	Probasics K3/K4 Wheelchair Stem Dust Cap - Front Fork	X1WCAS07
24	Probasics K3/K4 Wheelchair Fork Stem Bearings	X1WCAS08
25	Probasics K3/K4 Wheelchair Caster Axle Bolt w/Nut	X1WCAS09
26	Probasics K3/K4 Wheelchair Rear Wheel w/Bearings	X1WCAS10
27	Probasics K3/K4 Padded Armrest, Desk, Pair	X1WCAS20
28	Probasics K3/K4 Cross Brace, 16" - 18", Pair	X1WCAS22
29	Probasics K3/K4 Cross Brace, 20", Pair	X1WCAS23
ACCESSORY PARTS		
	Probasics K3/K4 Anti-Tippers	X1WCASANTI
	Probasics K3/K4 Elevating Leg Rest	ELR-C
	Probasics K3/K4 Swing Away Footrest	RSA-C