


PB42BARBED PART SCHEMATIC

Number	Available Item	Description
5	PBHB4PINX	BED INSERT PANEL (PR)
8	PBHB4HM	HEAD MOTOR
10	PBHB4HEAEXTCD	BARI EXT CORD HEAD MTR-JUNCTION BOX
13	PBHB4HLM	HI-LO MOTOR
15	PBHB4MCASTIN	CASTOR INSERT FOR HB4M
16	PBHB4MCASTWB	CASTOR WITH BRAKE
17	PBHB4FM	FOOT MOTOR
19	PBHB4JB	JUNCTION BOX FOR HB4
20	PBHB4PNDC	HAND RENDANT
25	PBHB4MCAST	CASTOR FOR HB4M
25	PBHB4SBAR	SIDE SUPPORT BARS
Not Shown	PBHB4SHAFT	BARIATRIC BED SHAFT 42" BED ONLY

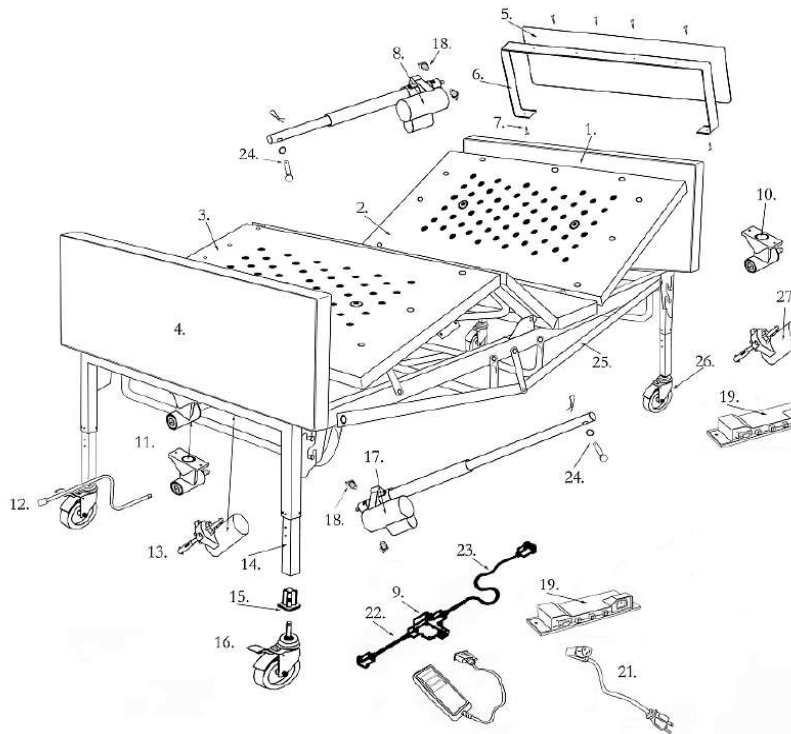

1. Complete headboard without wheels
2. Head end complete with electronics
3. Foot end complete with electronics
4. Complete footboard without wheels
5. Headboard composite wood panel - pair
6. Headboard/footboard frame
7. Panel frame screws, set of 6
8. Head end motor complete with extension tube and actuator
9. Headboard gear box
10. Head motor extension cord
11. Footboard gear box
12. Manual crank
13. Foot end hi-lo motor
14. Adjustable leg
15. Receptacle for caster
16. Caster with lock
17. Foot end motor complete with extension tube and actuator
18. Quick release spring knob
19. Junction box
20. Hand control
21. Power cord
22. Hi-lo shaft (two pieces)
23. Frame support rail
24. Attachment pin and hardware
25. Side support bars
26. Caster without lock