Euro Style Rollator

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RLAS10CB

RLAS10RG

RLAS10BK

RLAS10SH

RLAS10CM

RLAS10BW

# ROLLATOR DIAGRAM

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## WARNING

**THIS ROLLATOR HAS A WEIGHT CAPACITY OF 300 LB/136 KG. ALWAYS OBSERVE THE WEIGHT LIMIT ON THE LABELING OF YOUR ROLLATOR.**

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## INDICATIONS FOR USE

This device is intended to provide ambulatory assistance for end users with mild to moderate stability and minimal balance issues, and who can maintain good posture; the device handles are **NOT** intended to be used for weight bearing. User **MUST** be able to bear weight on both legs and both arms to use this device.

This device is **NOT** intended to be used as a transport device. Before operating the rollator at normal walking speed, practice maneuvering and making turns at slow rates of speed to get familiar with the rollator and how to best maintain balance. The seat of this device is intended to provide a temporary resting surface for people who tire easily and need frequent rest during walking.

This device is intended for both indoor and outdoor use.

## CONTRAINDICATIONS

The rollator is **NOT** intended for use by persons with severe equilibrium disorders or functional perceptual disorders.

The rollator is **NOT** intended for use by people with Alzheimer's disease or dementia, or those who are under the influence of drugs, sedated, confused or frail; these individuals are at increased risk of losing their balance and falls.

If you have issues with balance, weakness while standing, or need a firm immobile support to help you walk, you should **NOT** use a rollator and you should use a walker instead.

**SAVE THESE INSTRUCTIONS.** This sheet **MUST** accompany the device with each use.

## CAUTIONS

**READ ALL INSTRUCTIONS AND WARNINGS PRIOR TO ASSEMBLY AND USE. FAILURE TO USE THIS DEVICE AS INTENDED MAY INCREASE THE RISK OF INJURY TO THE USER OR DAMAGE TO THE DEVICE.**

- Dealers or resellers are responsible for providing this manual to any user or purchaser of this device. Failure to provide this manual to a user increase the risk of serious injury.
- If you **DO NOT** understand these instructions, contact a healthcare professional before attempting to use.
- Inspect all parts for shipping damages before use. If there is shipping damage – **DO NOT USE**. Contact the manufacturer or dealer for further instructions.
- If there are any problems with your rollator, **DO NOT** attempt to repair it yourself. Contact the manufacturer or dealer for any needed parts or repair instructions.
- Tires and brakes are non-durable components which will require maintenance throughout the life of the rollator.
- Brakes should be adjusted so that the wheels **DO NOT** move while the brake is applied – if brakes are unable to be adjusted correctly, **DO NOT** continue to use the rollator, seek repair or replacement.

 **CAUTION:** After making any adjustments to the rollator brakes, you **MUST** test the brakes before the rollator is used.

- Pay attention to any garments and/or accessories; **DO NOT** let clothing get caught between moving parts when folding or unfolding the rollator. Very long or loose pieces of clothing could also get caught between the wheels of the rollator, which could lead to serious injury or death.
- Tires should be inspected regularly for damage and wear and replaced when the tires are worn and show visible wear. Tires may wear faster if the device is used outdoors on rough surfaces.
- Before operating the rollator at normal walking speed, practice maneuvering and making turns at slow rates of speed to get familiar with the rollator and how to best maintain balance.

 **CAUTION:** Be conscious of potential pinch points during folding and unfolding of the device.

- Compass Health Brands assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.

## WARNINGS

- This rollator has a weight limit of 300 lbs. (136 kg.). **ALWAYS** observe the weight limit on the labeling of your rollator.
- **DO NOT** exceed 5 lbs. ( 2.267kg.) of weight inside pouch/ basket. **DO NOT** hang anything from the rollator handles or frame. Items may **ONLY** be carried in the provided pouch/basket.

- **DO NOT** remove warning stickers.
- **ALWAYS INSPECT ROLLATOR PRIOR TO USE.** Properly inspecting and maintaining your rollator can reduce the risk of injury. Make sure that all parts are secure and are in good working order. **ALWAYS** check the fasteners, wheels, brakes and seat for tightness prior to use. Make sure that the parking brakes are working and the rollator wheels cannot turn with the brakes on. If rollator is not in good working order or fasteners cannot be tightened, **DISCONTINUE USE OF THE ROLLATOR.**

- All wheels and handles **MUST** be adjusted to the same height for safe use. **DO NOT** use the rollator if wheels or handles are out of alignment or if hardware is not secure. As applicable to your rollator model, all hand knobs **MUST** be firmly tightened and height adjustment buttons **MUST** protrude fully through the height adjustment holes with an audible "click".
- **DO NOT** foot-propel the rollator while seated as this could cause imbalance and result in serious injury or death.

**DO NOT** use the rollator as a wheelchair. This is a walking aid **ONLY** and is **NOT** intended to be used as a transportation device. Improper use of the rollator can cause serious injury/death or damage to the device.

- **DO NOT** use rollator like a knee walker or scooter. **DO NOT** climb, kneel or stand on the rollator. This device should **ONLY** be used as a walking aid. When using this device **DO NOT** attempt to hop or perform any actions other than walking.
- **DO NOT** attempt to reach objects that are out of your immediate reach while seated on the rollator or while the rollator is in motion. **DO NOT** lean forward, backward or to either side while seated on the rollator. Ensure that you have come to a complete stop with the brake fully engaged or locked and have established good balance before reaching for an object.
- **ALWAYS** engage both parking brakes before sitting on the rollator. **DO NOT** move or scoot while seated in the rollator. Parking brakes **MUST ALWAYS** remain engaged while seated.
- When sitting on the rollator, keep both feet in contact with the floor and **DO NOT** lean out of the seat.
- **DO NOT** use rollator with **ONLY** one upper limb (arm/hand) or one lower (leg/foot) limb at a time as this can cause serious injury or death.
- **ALWAYS** wear protective footwear while using the rollator. **ALWAYS** keep shoelaces, loose clothing and/or accessories clear of the wheels and handlebars during use.

- **DO NOT** make any adjustments to the rollator while it is in use.
- **ALWAYS** maintain good posture while using the rollator. When walking with the rollator, keep your feet in line with the rear wheels and **DO NOT** let the rollator get too far in front of you.
- **DO NOT** use the rollator to support your weight as you stand, otherwise serious injury could occur.
- **DO NOT** place arms or elbows on handles to walk or rise from seated position.
- **DO NOT** sit on the rollator while it is being loaded into a vehicle or while it is being transported in a vehicle.
- **DO NOT** use the backrest bar to support your weight. Leaning back while seated on the rollator may cause the device to tip and could result in serious injury or death.
- **ACCESSORIES WARNING** Compass Health Brands Corp. products are specifically designed and manufactured for use in conjunction with Compass Health Brands accessories. Accessories designed by other manufacturers have not been tested by Compass Health Brands and are not recommended for use with Compass Health Brands products.
- **ALWAYS** have all wheels in contact with the floor/ground while using this device. Failure to maintain wheel contact could result in improper balance of the rollator and cause injury or damage.
- **DO NOT** sit on the rollator if it is parked on a slope. **ONLY** use the seat on level ground.
- **ALWAYS** make sure the rollator is fully opened and the folding bar is locked in the down position before use.
- This is not a toy. **DO NOT** let children play on the rollator.
- **BE AWARE** of your surroundings when operating the rollator. Look for hazards on the ground and avoid them to prevent falls. Avoid streets and surfaces with water, sand, gravel, dirt, leaves and other debris. Wet, slick, uneven and/or rough surfaces may impair traction and contribute to possible accidents. Loose cords and unsecured rugs may move suddenly and cause a loss of balance. **DO NOT** use a rollator in mud, ice or puddles.
- **ALWAYS** take additional care and move slowly when moving from carpeted to hard surface floors or cracks/separations in walkway surfaces; avoid sharp bumps, drainage grates and sudden surface changes. The rollator may abruptly stop if a wheel becomes wedged.
- **NEVER** use on or near steps, sloped driveways/hills with a steep incline (**NEVER** greater than 15 degrees) as there

is a risk of losing balance. **DO NOT** walk backwards down a slope. **ONLY** mount or dismount your rollator on a level surface.

- **DO NOT** use when going up or down stairs or escalators, or the moving walkway/autowalk/people mover.
- **DO NOT** use near swimming pool areas.
- **DO NOT** use the hand brake as a primary stopping tool. It may be used if needed but is primarily designed to maintain a stopped stance.
- **DO NOT** use while under the influence of mind-altering drugs, alcohol or while suffering from dizziness.
- **DO NOT** store in direct sunlight or outdoors; exposing this device to the elements could cause corrosion-like damage to the device which can lead to serious injury or death. Exposure to extreme temperatures (heat, cold, moisture, etc.) may make surfaces (seat, hand grips, etc.) hot to the touch, weak or brittle or cause advanced corrosion. Ensure hand grips **DO NOT** twist and skin contact surfaces are cool to the touch before use.

**FAILURE TO USE COMMON SENSE AND HEED THE ABOVE WARNINGS FURTHER INCREASES RISK OF SERIOUS INJURY OR DEATH. USE AT YOUR OWN RISK AND WITH APPROPRIATE AND SERIOUS ATTENTION TO SAFE OPERATION. USE CAUTION.**

## ASSEMBLY INSTRUCTIONS

### **WARNING**

Inspect all parts for shipping damages before use. If there is shipping damage—**DO NOT USE**. Contact the dealer/distributor immediately for further instructions. If there are any problems with the rollator, **DO NOT** attempt to repair it yourself. Contact the dealer/distributor that you purchased the rollator from for any needed parts or repairs.

Compass Health Brands assumes no responsibilities for any damage or injury caused by improper installation, assembly, or use of this product.

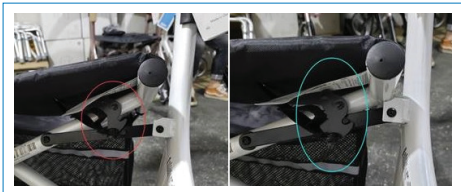
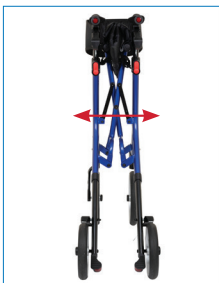
### **UNFOLDING ROLLATOR**

1. Remove rollator from its packaging and place it so all four wheels contact the floor.
2. While gripping handlebars, push outward until rollator is fully open.

- Once rollator is open, push firmly on both sides of the seat to flatten it.
- When the seat is completely flat, an audible click will be heard.

## **WARNING**

Ensure the rollator is fully opened, the seat is snapped down. To ensure that the seat is properly assembled for use, place **BOTH** hands on the opposite sides of the seat frame. Push down until you hear and audible click, letting you know the rollator is ready for operation. Please also ensure the backrest is locked and fully secured **BEFORE** use. See below images for reference.

<b>CAUTION</b>	
<b>Pinch Point</b> Be cautious of potential pinch points during folding and unfolding of the device.	
<b>PRUDENCE</b>	
<b>Point de pincement</b> Faites attention aux points de pincement potentiels lors du pliage et du dépliage de l'appareil.	
<b>PRECAUCIÓN</b>	
<b>Punto de pellizco</b> Tenga cuidado con los posibles puntos de pellizco al plegar y desplegar el dispositivo.	

**NOTE:** Keep hands and fingers clear of moving parts to avoid injury.

## SETTING SEAT HEIGHT ADJUSTMENT

- Push detent button located outside the rear leg.
- While depressing the button, pull the leg outward to increase the seat-to-floor height. Push the leg inward to decrease the seat-to-floor height.
- Release the button and ensure the detent button fully protrudes from the corresponding hole.

4. Repeat steps 1 through 3 for the other legs so all four legs are at the equal and desired height. All settings **MUST** be at the same setting to ensure stability and that all four wheels are touching the ground.

## SETTING HANDLE HEIGHT ADJUSTMENT

1. Push red height release button on back of the handle.
2. While depressing the button, pull the handles upward.
3. Move the handles upward until the desired height is reached.
4. Release the button and ensure the button snaps into place at the corresponding hole.
5. Repeat steps 1 through 4 for the second handle so both handles are at an equal and desired height.

### **WARNING**

All wheels and handles **MUST** be adjusted to the same height for safe use. **DO NOT** use the rollator if wheels or handles are out of alignment, or if hardware is not secure. As applicable to your rollator model, all hand knobs **MUST** be firmly tightened and height adjustment buttons **MUST** protrude fully through the height adjustment holes with an audible “click”.

## ATTACHING FRONT POUCH

1. Remove pouch from packaging.
2. Unfold rollator (refer to Unfolding Rollator section on pg. 7 & 8) and unfold pouch by releasing hook and loop straps.
3. Slide the loops on each side of the pouch over the knobs extending from the front of the rollator’s seat.
4. Slide pouch all the way onto knobs until the pouch is firmly in place.

## CANE HOLDER INSTALLATION

1. Adjust lower cane platform and upper cane holder tip to desired setting using a 4 mm Allen wrench.

Upper Clip

Lower Clip

2. After both attachments have been adjusted properly, place the upper clip directly under the handle height adjustment mechanism.

3. Place the cane holder platform on lowest available section on the frame (the colored section), ensuring that it does not interfere with the position of the detent button.

**NOTE: IF THE SEAT HEIGHT IS SET TO THE HIGHEST SETTING, PLACE THE CANE HOLDER PLATFORM JUST ABOVE THE DETENT BUTTON.**

4. Rest cane tip in bottom platform and clip the cane shaft into the upper clip.

## **WARNING**

**ALWAYS** ensure that the cane tip is securely seated in the lower platform to prevent a tripping or fall hazard that could cause serious injury.

**ALWAYS** use **CAUTION** when cane attachment is in use to prevent the cane grip or wrist strap from getting caught or snagged on nearby objects.

### **BRAKE OPERATION**

1. To engage the parking brake (lock the brakes), press down on the brake handle until the handle locks in place.
2. To release and/or engage the brakes, squeeze the brake handle up towards the hand grip.

## **WARNING**

**DO NOT** use the hand brake as a primary stopping tool. It may be used if needed but is primarily designed to maintain a stopped stance.

Brakes and tires are non-durable components which will require maintenance through the life of the rollator. Brakes **MUST** be adjusted so that the wheels do not turn while the brake is applied or when the brake lock/parking brake is engaged. If wheels turn while the brake is applied and/or when the brake lock is engaged, **DO NOT** continue to use the rollator. If you are unable to adjust, repair or replace, contact your dealer/distributor for assistance.

### **BRAKE ADJUSTMENT**

1. Hold the screw housing firmly.
2. Adjust brake by pressing downwards on the screw housing and turning clockwise or counterclockwise to adjust. Refer to images on page 9 for further assistance.

## FOLDING/STORAGE

1. Ensure items in the pouch have been removed.  
The rollator can easily fold from the user's side by pulling the nylon strap on the seat up. The rollator will begin to fold as the nylon strap is pulled.

## WALKING WITH THE ROLLATOR

1. Standing behind the rollator, slowly roll forward so that the rear wheels are a few inches **IN FRONT** of your feet.
2. Step forward and place your foot **IN LINE** with the rear wheels, but slightly inside of the wheels.
3. Maintaining a steady roll forward, place your other foot **IN LINE** with the rear wheels.
4. Determine a safe gait to keep your feet **IN LINE** with the rear wheels.

### CAUTION

- **BE AWARE** of your surroundings when operating the rollator. Look for hazards and avoid them. Avoid streets and surfaces with water, sand, gravel, dirt, leaves, and other debris. Wet, slick, uneven, and/or rough surfaces may impair traction and contribute to possible accidents. Loose cords and unsecured rugs may move suddenly and cause a loss of balance.

# CLEANING AND MAINTENANCE

1. Clean the rollator using mild soap and water. Wipe dry.
2. Your rollator should be checked periodically to ensure the brakes are functioning properly and that all nuts and bolts are secure.
3. Check that the front castor wheels swivel freely. **DO NOT** use the rollator if they **DO NOT**.
4. Periodically test the brakes by both walking and squeezing the brakes and by stopping, pushing down on the brakes to a locked position and then trying to push the walker. If the brakes **DO NOT** lock the wheels, the brakes need to be adjusted. Refer to page 11 for brake adjustment instructions. If adjustment does not tighten the brakes, **DO NOT** use the rollator.
5. The expected product life cycle of this product is approximately 5 years when used in accordance with safety instructions, maintenance intervals and correct use, stated in this manual. The effective product life cycle can vary according to frequency and intensity of use.

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# LIMITED LIFETIME WARRANTY

This warranty is solely for the benefit of the original consumer purchaser.

This device was built to exacting standards and was carefully inspected prior to shipment. Your ProBasics® brand product is warranted to be free of defects in materials and workmanship, has a Limited Lifetime Warranty on the frame and a one year warranty from date of purchase on all other parts, which are subject to normal wear and need periodic replacement. If within such warranty period, the product proves defective, the product will be repaired or replaced, at the option of Compass Health Brands Corp. This warranty does not cover product failures due to normal wear and tear or device misuse.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty provides only for replacement of defective parts and does not cover shipping or labor charges. Compass Health Brands' sole obligation and your exclusive remedy under this warranty is limited to such repair and/or replacement.

This warranty shall not apply to serial numbered products if the serial number has been removed or defaced or to defects or damages caused by unauthorized parts, attachments or repairs, negligence, accident, circumstances beyond Compass Health Brands' control, or improper operation, maintenance or storage.

If you have a question about your Compass Health Brands product or this warranty, please contact an authorized Compass Health Brands Dealer.

Compass Health Brands assumes NO responsibility for any damage or injury caused by improper installation, assembly, maintenance or use of this product.

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RLAS10\_Euro\_Rollator\_HT\_05\_240314 ©2024