Transport Chair

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OPERATING INSTRUCTIONS

Transport wheelchairs are lightweight mobility chairs that combine the function of a wheelchair with the portability of a rollator. They are meant to be pushed by a family member or caregiver, as their small wheels make it impossible for the user to propel themselves.

It is important for the family member or caregiver to be familiar with the proper assisting procedures in any situation that requires the use of the transport chair. Safety is the key consideration. Consult your physician, nurse or physical therapist to determine methods most suitable for your individual needs and abilities. Your homecare dealer can assist you in making adjustments and provide you with operational information.

NOTE: The transport chair should not be operated without the assistance of a family member or caregiver.

TO OPEN CHAIR

Remove all packaging including zip ties, bubble wrap and plastic from the wheels before use.

1. Standing next to the transport chair, grasp the armrest closest to you ⑩.
2. Lean the transport chair toward your body, raising the opposite wheels off the ground/floor.
3. Push the other armrest ⑩ away from you to start unfolding the transport chair. Return all wheels to the ground/floor. Press downward on the seat rails (where the seat upholstery is attached to the frame) on both sides, until the transport chair is fully open.

BACK SUPPORT

1. Raise Back Support

Once the chair has been opened and secured, grasp the two (2) hand grips ① and lift the back support to an upright position. Once fully upright, the back support locks ② will engage automatically making an audible "click" on each side.

2. Lower Back Support

To fold the back support down for storage or transportation, push up on the back support lock levers ② simultaneously to unlock each side. Carefully fold down the back support.

SWING-AWAY FOOTRESTS

How To Attach

4. Position a footrest so it is pointing 90 degrees away from the transport chair (footrest platform ⑦ should be pointing forward when folded down at this point).
5. Align the holes with the footrest pegs ④ on the frame and gently press down until the footrest can swing freely.
6. Swing the footrest until the footrest platform ⑦ is pointing inward. Continue with this rotation until the footrest locks in place ⑧.
7. Repeat steps 1 – 3 with the other footrest.

How To Detach

1. Release the footrest lock ⑧ and swing the footrest to the side.
2. Gently pull up on the footrest and remove it from the footrest pegs ④.
3. Repeat steps 1 – 2 with the other footrest.

How To Adjust the Length

1. Have the user sit in the transport chair with their feet on the footrest platform.
2. Unlock the Height Adjustment Safety Lock ⑥, press the height adjustment push buttons ⑤, then raise or lower footrests to desired position that will provide maximum support of the users thighs.

If there is an uncomfortable pressure being applied to the back of the users thighs, then the footrests are too low. Footrest should not be lower than 2½" from the ground to ensure proper clearance. When transferring, please ensure that footrests locks are disengaged for user safety.

WHEEL LOCKS

NOTE: It is extremely important to engage both wheel locks before getting in or out of the transport chair, while in an elevator or on a wheelchair lift.

Proper adjustment for the wheel lock is to depress into the wheel at least ⅛" when locked.

Locks are engaged by pushing each handle completely forward ③. Adjustments to locks are made by loosening the bolt on the frame, sliding the lock housing to the desired position and tightening the bolt again (**DO NOT** over tighten).

STABILITY

Transport chairs are designed to be safe and stable as long as the center of gravity is maintained. Care must be taken while performing activities that shift your center of gravity while in the chair.

WARNINGS

DO NOT ATTEMPT TO SET-UP, ADJUST OR OPERATE THIS TRANSPORT CHAIR WITHOUT READING ALL INSTRUCTIONS AND WARNINGS CAREFULLY.

- **A FAMILY MEMBER/CAREGIVER (ATTENDANT) MUST BE PRESENT AT ALL TIMES WHILE THE TRANSPORT WHEELCHAIR IS IN USE. NEVER LEAVE THE OCCUPANT UNATTENDED.**
- **NOTE:** You and your attendant should consult a qualified professional for correct techniques on ramp negotiations.
- Any person assisting the rider in maneuvering or lifting of the wheelchair must have sufficient strength and be able to see obstacles in the path of travel.
- **ALWAYS INSPECT PRIOR TO RIDING.** Properly inspecting and maintaining your transport chair can reduce the risk of injury. Make sure that all parts are secure and are in good working order. Always check the fasteners, wheels, wheel locks and seat for tightness prior to use. If it is not in good working order or fasteners cannot be tightened, **DISCONTINUE USE OF THE TRANSPORT CHAIR.**
- The transport chair is intended for use with a single rider. **ALL WHEELS** must be in contact with the floor/ground at **ALL TIMES**. Failure to maintain this could result in improper balance of the chair and cause injury or damage.
- Be aware of your surroundings when pushing the transport chair. Look for hazards and avoid them. Avoid streets and surfaces with water, sand, gravel, dirt, leaves, and other debris. Wet, slick, uneven and/or rough surfaces may impair traction and contribute to possible accidents. **DO NOT** push the transport chair in mud, ice, puddles or water.
- To determine and establish your particular safety limits, practice bending, reaching and transferring activities in several combinations in the presence of a qualified health professional before attempting active use of the transport chair.
- Make sure the back is locked securely before using this transport chair.

- **ALWAYS** check hand grips for looseness before using the transport chair. If loose and/or worn, replace immediately.
- **DO NOT** use an escalator to move a transport chair between floors. Serious bodily injury may occur.
- **NEVER** leave an unoccupied transport chair on an incline.
- **DO NOT** attempt to stop a moving transport chair with wheel locks. Wheel locks are not brakes.
- **ALWAYS** lock both wheel locks before transferring to and from the transport chair.
- **DO NOT** sit or transfer into the transport chair unless it is fully open.
- **DO NOT** attempt to move up or down an incline with water, ice or oil film.
- **DO NOT** attempt to lift the transport chair by any removable (detachable) parts. Doing so may result in injury to the user or damage to the transport chair.
- Never use near steps, sloped driveways/hills with an incline of more than 10 degrees or swimming pool areas.
- **DO NOT** attempt to reach objects that are out of your immediate reach while using this device. **DO NOT** reach for objects if you have to lean or shift position on the transport chair. **DO NOT** lean forward, backward or to either side while sitting on the transport chair. This may cause tipping.
- To prevent tipping while seated, **DO NOT** push yourself with your feet.
- **DO NOT** step on the foot plates when transferring, as this could cause tipping.
- **DO NOT** attempt to reach objects if you have to move forward in the seat.
- **DO NOT** attempt to reach objects if you have to pick them up from the floor by reaching down between your knees.
- **DO NOT** lean forward out of the wheelchair any farther than the length of the armrests.
- **DO NOT** tilt the transport chair without assistance.
- **DO NOT** attempt to ride over curbs or obstacles. Doing so may cause the transport chair to tip over and cause bodily harm or damage to the transport chair.
- **DO NOT** transfer in or out of the transport chair on your own unless you have sufficient upper body strength, balance and agility and have been instructed in transfer techniques by a health professional.
- Use caution when moving from carpeted to hard surface floors or cracks/separations in walkway surfaces; avoid sharp bumps, drainage grates, and sudden surface changes. The transport chair may abruptly stop.

- Never make a sharp turn at high speed; do not turn at speeds over 1 mph (very slow walking speed).
- Compass Health Brands products are specifically designed and manufactured for use with Compass Health Brands replacement parts. Parts designed by other manufacturers have not been tested by Compass Health Brands and are not recommended for use with Compass Health Brands products.
- This is not a toy. **DO NOT** let children play on the transport chair.
- **DO NOT** store in direct sunlight or outdoors. Exposure to extreme temperatures (heat, cold, moisture, etc.) may make surfaces (seat, arm rests, hand grips, etc.) hot to the touch, weak or brittle. Ensure hand grips **DO NOT** twist, and skin contact surfaces are cool to the touch before use.
- **ALWAYS** keep both hands inside the transport chair when it is in motion.
- Keep shoelaces, loose clothing and/or accessories out of the way of the wheels and other moving parts.
- **ALWAYS** keep hands and fingers clear of moving parts to avoid injury.
- Wash the upholstery at least once a month with soapy water. Confirm that the upholstery is dry before using.
- The upholstery is secured to the frame with bolts. Always inspect that the upholstery is in good condition before use. If it is ripped, torn or cannot be secured, **DISCONTINUE USE OF THE TRANSPORT CHAIR.**
- Compass Health Brands does not recommend the use of its transport chairs during a weight training session.

**FAILURE TO USE COMMON SENSE AND
HEED THE ABOVE WARNINGS FURTHER
INCREASES RISK OF SERIOUS INJURY.
USE AT YOUR OWN RISK AND WITH
APPROPRIATE AND SERIOUS ATTENTION
TO SAFE OPERATION. USE CAUTION.**

GENERAL CARE

Proper care and maintenance is important in order to keep this transport chair in good working condition. Regular cleaning and inspection for loose or worn parts is recommended. Follow these maintenance procedures to help extend the life of this transport chair. A qualified dealer or repair center should be consulted for periodic inspections.

FRAME

Wipe chrome parts with a clean, soft cloth at least once a week. All metal/aluminum parts can be cleaned with auto wax or similar products. If exposed to moisture, dry immediately. Do not use any abrasive cleansers, they will scratch and damage the paint & chrome the finish.

TIRES

All tires should be inspected at least once a month to ensure proper tightness. If a wheel wobbles or is loose, release the lock nut and tighten the axle bolt slightly. Test for proper tightness and alignment by leaning the chair to one side and spinning the adjusted wheel. Solid rubber tires need minimal maintenance, but should be cleaned occasionally with a damp cloth. If any of the tires become severely worn or cracking appears, **STOP USE** and replace immediately.

SEAT UPHOLSTERY

Wash the upholstery at least once a month with soapy water. Confirm that the upholstery is dry before using. If worn or torn upholstery is discovered, **STOP USE** and replace immediately as it may not support the riders body weight.

HAND GRIPS

Wipe the hand grips with cold water and a clean towel. **NEVER** use ammonia-based cleaning solutions or soap on the hand grips. This may cause them to loosen or slide off the frame. Before using the transport chair again, confirm that the grips are completely dry.

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LIMITED LIFETIME WARRANTY

This warranty is solely for the benefit of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. ProBasics Brand Wheelchair carries a limited lifetime warranty on the frame and 1 year on all other non-durable components. If within such warranty period, the product proves defective, the product will be repaired or replaced, at the option of Compass Health Brands. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear.

This warranty shall not apply to serial numbered products if the serial number has been removed or defaced, or to defects or damages caused by unauthorized parts, attachments or repairs, negligence, accident, or any circumstance beyond Compass Health Brands control, such as improper operation, maintenance or storage.

If you have a question about your Compass Health Brands product or this warranty, please contact an authorized Compass Health Brands Dealer.

 **COMPASSHEALTH**

6753 Engle Road
Middleburg Heights, Ohio 44130
Tel.: (800)376-7263
www.compasshealthbrands.com
42-TCA1916_TCS1916_00 ©2017