K4 Deluxe Wheelchair

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INDICATIONS FOR USE

The device is intended for medical purpose to provide mobility to persons restricted to a sitting position.

CONTRAINDICATIONS

- This device **IS NOT** intended for visually impaired people.
- This wheelchair is **NOT** intended for use without supervision by people with Alzheimer's disease or dementia or those who are under the influence of drugs, sedated, confused or frail; these individuals are at increased risk of losing their balance and falls.
- Users with limited physical capabilities should be supervised or assisted when transferring in/out of wheelchair or when using ramps.

WARNINGS

DO NOT ATTEMPT TO ADJUST OR OPERATE WHEELCHAIR WITHOUT READING ALL INSTRUCTIONS CAREFULLY. COMPASS HEALTH BRANDS CORP. ASSUMES NO RESPONSIBILITY FOR ANY DAMAGE OR INJURY CAUSED BY IMPROPER INSTALLATION, ASSEMBLY OR USE OF THIS PRODUCT. THIS PRODUCT SHOULD NOT BE USED WITHOUT INSTRUCTION FROM A HEALTHCARE PROFESSIONAL. FAILURE TO USE THIS DEVICE AS INTENDED MAY INCREASE THE RISK OF INJURY TO THE USER OR DAMAGE TO THE DEVICE.

THIS ITEM SHOULD NEVER BE SECURED FOR TRANSPORT IN A MOVING VEHICLE. ALWAYS TRANSFER WHEELCHAIR USER TO A VEHICLE MANUFACTURER'S APPROVED SEAT AND USE SAFETY RESTRAINTS WHEN AVAILABLE.

- This wheelchair has a weight limit of 300 lbs. (136kg.). **ALWAYS** observe the weight limit on the labeling of your wheelchair.

- Dealers or resellers are responsible for providing this manual to any user or purchaser of this device. Failure to provide this manual to a user increases the risk of serious injury.

BEFORE EACH USE:

- Make sure the chair rolls easily and that all parts work smoothly.
- Check for noise, vibration or any change in use. Changes may indicate loose parts or wheelchair damage.
- Check to ensure wheel locks are in good working order.
- Make sure hand grips are **NOT** loose and cannot be moved or twisted.
- Make sure the seatback is locked in the upright position
- Make sure that the lap positioning belt (if applicable) is securely attached to the wheelchair.
- If the wheelchair is damaged, discontinue use and contact your dealer/distributor for authorized parts, repairs or replacement.
- If there are any problems with your wheelchair, **DO NOT** attempt to repair it yourself. Contact the dealer/provider or store for any needed parts or repair.
- WARNING!** Each wheelchair is a unique piece of medical equipment. Whether you are a new wheelchair user or have years of experience, you **MUST** take the time to acclimate to the wheelchair before you begin riding. Start slowly and take the time to learn the handling, maneuvering and ride characteristics of this wheelchair.

WARNING: There are potential finger pinch point areas at moving parts and between the seat frame and the seat cradles on some models. When opening the wheelchair, take care **NOT** to place your hand or fingers between the seat frame and the seat cradles (if present) or on any other moving parts. **DO NOT** place your hand or fingers on the underside of the seat frame rail

when opening or closing the wheelchair. **DO NOT** sit in the wheelchair until the seat is completely in the open position. The risk of injury may occur if care is **NOT** taken.

- DO NOT** place your hands between the seat rail and side panel.
- DO NOT** sit or transfer into the wheelchair unless it is fully open.
- ALWAYS** learn as much as you can about the places where you will be using your wheelchair before you get there. **ALWAYS** be alert for hazards and learn how to avoid them.
- Be aware of your center of gravity when you first start using your wheelchair.

- Be careful and think of ways to prevent problems before they occur. Plan for emergencies such as brake failure, tipping backward or the wheelchair moving by itself.
- **DO NOT** self-propel using your feet. To prevent injury or harm, feet must be on the footrests while moving.
- **DO NOT** operate your wheelchair if your physical limitation impairs your ability to operate the wheelchair safely. Examples include, but are not limited to, a lack of awareness of surroundings or lack of capacity to think clearly, impaired vision or hearing or impaired dexterity of upper extremities that limit the capacity to use controls or steer. Operating this wheelchair while impaired may result in erratic steering, acceleration, braking, instability, tipping, misdirection and/or collision with persons or objects. Any of these circumstances can result in serious personal injury or death.
- Consult with your physician if you are taking prescribed or over-the-counter drugs/medications. Some drugs/medications may impair your ability to operate your wheelchair safely. Operating this wheelchair while impaired may result in erratic steering, acceleration, braking, instability, tipping, misdirection and/or collision with persons or objects. Any of these circumstances can result in serious personal injury or death.
- It is **NOT** recommended that the wheelchair be operated on roadways, streets or any surfaces with vehicular traffic. If necessary to operate the wheelchair on a roadway or street, use flashy items such as flashing taillights and flags. Use headlights and flashing taillights if you ride in the streets at night. Pick bright colored wheelchairs.
- Obey all local pedestrian traffic rules. Wait until your path is clear of traffic and proceed with extreme caution. It is recommended that you use safety equipment such as reflectors/reflective clothing, lights and safety flags.
- At nighttime and in other low light or low visibility settings, it may be difficult for you to see the path ahead or your surroundings to avoid dangerous surfaces or collision with obstacles, vehicles or pedestrians. **NEVER** proceed when you are unable to clearly see around you and the pathway ahead. Use of additional lights or lighting accessories may enhance your ability to see around you and ahead. Failure to use these safety features may result in severe injury, property damage or death. Contact your authorized provider for information on the availability of lighting accessories.
- **DO NOT** attempt to operate on inclines 10° or greater without proper assistance.
- **DO NOT** use wheel locks to slow your descent down an incline. Abruptly changing directions while going down an incline could cause instability or tipping.

- Ramp surfaces may become slippery when wet. Exercise extreme caution when using a ramp during inclement weather conditions. Failure to heed this warning can result in personal injury and/or property damage.
- **ALWAYS** avoid unsafe conditions, such as soft, slippery surfaces, uneven paths and obstacles. Avoid water, icy surfaces, sand, gravel, loose dirt, leaves and other debris. Wet, slick, uneven and/or rough surfaces may impair traction and contribute to accidents.
- If you feel unsure about a driving surface, avoid that surface.
- This wheelchair is **NOT** intended for use on escalators or stairs. **DO NOT** carry an occupied wheelchair up or down a stairway. Serious bodily injury may occur.
- When cornering sharply, reduce your speed and maintain a stable center of gravity. Avoid shifting your weight in the opposite direction of the turn, which may cause the wheelchair to lose traction and tip over.
- **DO NOT** attempt to ride over curbs or obstacles. Doing so may cause your wheelchair to tip over and cause bodily harm or damage to the wheelchair. Consider reducing speed in half when beginning to turn the wheelchair. This will help to ensure user safety.
- Avoid tall grass that can entangle the running gear.
- Keep loose objects or lap cover away from the wheel spokes. Keep your hands, clothing and all other objects away from the wheels when operating your wheelchair. **DO NOT** allow any personal belongings or objects to drag behind the wheelchair when operating. Be aware that loose fitting clothing or other objects can become caught in all moving parts of the wheelchair. Failure to heed this may result in personal injury and/or property damage.
- Ensure that any oxygen or air tubing or other accessories are correctly routed and secured so that they **DO NOT** become entangled, kinked or severed.
- **DO NOT** allow objects or accessories to protrude from your wheelchair when operating. Be aware protruding objects can become caught on items surrounding the wheelchair's path.
- Ensure wheels are locked when using a cell phone or other personal items.
- **ALWAYS** completely engage wheel locks when getting in and out of wheelchair or when in elevator or wheelchair lift.
- Wheelchair should be on a smooth, stable and level area with wheel locks engaged before transferring user in and out of wheelchair.
- When getting in or out of wheelchair, **DO NOT** stand on or apply weight to footrests. Footrests **MUST** be flipped up/out of the way before attempting to stand/walk.
- **DO NOT** adjust any footrests lower than 2½" from floor.

- **DO NOT** lean on or apply weight to the top of the wheelchair back during transfer. This can cause the wheelchair to tip over. When trying to extend arms over back, only reach as far as your arm can extend without having to alter your seating position or center of gravity.
- Before reaching or leaning forward, rotate casters to the front of the wheelchair and engage wheel locks to help prevent wheelchair from tipping over.
- **DO NOT** carry passengers on your wheelchair. This wheelchair has been designed to accommodate one individual. If more than one individual occupies the wheelchair this may adversely affect the stability of the wheelchair, resulting in serious bodily injury to the user and passenger and damage to the wheelchair and surrounding property.
- **DO NOT** stand or kneel on wheelchair seat. **DO NOT** stand on the frame of the wheelchair.
- **DO NOT** use your wheelchair on or near railroad tracks or crossings.
- Compass Health Brands Corp. does **NOT** recommend the use of a wheelchair as a weight training apparatus. The wheelchairs have **NOT** been designed or tested as a seat for any kind of weight training. If the occupant uses said wheelchair as a weight training apparatus, Compass Health Brands Corp. shall **NOT** be liable for bodily injury and the warranty is void.
- **DO NOT** use the wheelchair to haul, move or tow other vehicles, including lawn mowers, ATV equipment or other mobility devices. Your wheelchair is **NOT** designed with such use in mind and any damage or injury incurred from such use is **NOT** the responsibility of the wheelchair manufacturer.
- This wheelchair is **NOT** designed for use in pools, showers, bathtub, sauna or other water environments; this could result in corrosion of parts that result in damage to the wheelchair and possible injury to the user. **USE EXTREME CAUTION** if you use your wheelchair near or in bodies of water.
- **DO NOT** store in direct sunlight or outdoors. Exposure to extreme temperatures (heat, cold, moisture, etc.) may make surfaces hot to the touch, weak or brittle. Ensure hand grips **DO NOT** twist and skin contact surfaces are cool to the touch before use.
- **DO NOT** attempt to lift the wheelchair by any movable or detachable parts. Lifting by means of any movable or detachable parts of the wheelchair may result in serious injury to the user or damage to the wheelchair.
- **NEVER** transport a wheelchair in the front seat of a vehicle **UNLESS** it is secured in place, as it could move and interfere with the driver.

- **NEVER** use any chair that has been involved in a motor vehicle accident. A wheelchair that has been involved in a motor vehicle accident may be damaged in ways that are not readily apparent and which could cause the chair to fail in use.
- **DO NOT** tie down or attach anything to the wheels. This could cause tipping and possibly result in injury or damage to the wheelchair.
- **DO NOT** attempt to perform a “wheelie” (tilting the wheelchair backward until it reaches its balance point) in your wheelchair because of the dangerous nature of this kind of maneuver. Doing a “wheelie” is dangerous and could result in personal injury to the user.
- This is not a toy. **DO NOT** let children play with or around the wheelchair. They should be instructed **NEVER** to touch your wheelchair and to **ALWAYS** keep hands and feet away from wheels and other moving parts.
- **DO NOT** modify the wheelchair. Examples: drilling holes, grinding, welding parts, using non-authorized / non-approved parts. Such modifications may change the chair’s structure and center of balance which may increase the risk of serious injury to the user and void the warranty.
- Accessories designed by other manufacturers have **NOT** been tested by Compass Health Brands Corp. and are **NOT** recommended for use with this product.

ACCESSORIES WARNING: Compass Health Brands Corp. products are specifically designed and manufactured for use in conjunction with Compass Health Brands accessories. Accessories designed by other manufacturers have **NOT** been tested by Compass Health Brands and are **NOT** recommended for use with Compass Health Brands products.

- **WARNING:** When you drive your wheelchair on a slope in climbing condition, transfer the gravity of the wheelchair to front by leaning your body forward to prevent the front wheels from lifting.

FAILURE TO USE COMMON SENSE AND HEED THE ABOVE WARNINGS FURTHER INCREASES RISK OF SERIOUS INJURY. USE AT YOUR OWN RISK AND WITH APPROPRIATE AND SERIOUS ATTENTION TO SAFE OPERATION.

ASSEMBLY INSTRUCTIONS

Remove the wheelchair from all packaging including tape, cardboard and plastic. Check all parts of the wheelchair for damage before use. If any parts are missing or damaged, **DO NOT** attempt to use the wheelchair. If you do not understand the manual, contact your dealer for a complete explanation of the instructions, cautions and warnings. Otherwise injury may result. Dealers or resellers are responsible for providing this manual to any user or purchaser of this device. Failure to provide this manual to a user increases the risk of serious injury.

BEFORE EACH USE:

- Make sure the chair rolls easily and that all parts work smoothly.
- Check for noise, vibration or any change in use. Changes may indicate loose parts or wheelchair damage.
- Check to ensure wheel locks are in good working order.
- Make sure hand grips are **NOT** loose and cannot be moved or twisted.
- Make sure the seatback is locked in the upright position.
- Make sure that the lap positioning belt (if applicable) is securely attached to the wheelchair.
- If the wheelchair is damaged, discontinue use and contact your dealer/distributor for authorized parts, repairs or replacement.

CAUTION: DO NOT use excessive force when installing any parts.

TO OPEN/UNFOLD WHEELCHAIR

Open wheelchair by tilting to one side toward you and pushing down on seat rail until fully opened. Ensure seat rails are fully cradled in seat rail guides (Fig. A).

TO CLOSE/FOLD WHEELCHAIR

First, fold foot plates to vertical position. With both hands, grasp the middle of the seat upholstery at the front and back edge and lift up. Or, tilt chair to one side toward you and lift up on seat rail.

CAUTION: Keep hands and fingers clear of moving parts to avoid injury.

INSTALLING/REMOVING THE FRONT RIGGINGS

1. Turn the front rigging to the side (open footplate is perpendicular to wheelchair).
2. Install the front rigging hinge plates onto the hinge pins on the wheelchair frame (Fig. B).
3. Push the front rigging towards the inside of the wheelchair until it locks into place. An audible click will be heard.

NOTE: The footplate will be on the inside of the wheelchair when locked in place.

4. Repeat steps 1-3 for the opposite side of the wheelchair.
5. To release the front rigging, push the footrest release lever inward, rotate footrest outward. To remove the front rigging, lift and remove from hinge pins.

ANTI-TIPPER ADJUSTMENT

Anti-Tippers will come preinstalled on your K4 Deluxe out of the box.

1. To adjust the height of the anti-tippers. Squeeze and push the red buttons forward. With your other hand, you will be able to

freely move the anti-tipper up and down to the desired height. Once you find the correct height setting, release the red buttons. Ensure that the anti-tipper is locked in place. You will hear an audible click once it is locked into place.

ATTACHING 2" SEAT EXTENDER

1. Remove tube end caps and U-clips from either side of the wheelchair seat rails.
2. Take the seat extender attachment with hook and loop side up and align into corresponding seat rail tubes. Line up the holes on the seat rails to the holes on the seat extension tubes.
3. Insert the U-clips and depress until fully locked onto the seat rail.
4. Slide the hook and loop upholstery extension under the seat upholstery on the wheelchair to adhere to the matching hook and loop strip.

NOTE: Ensure the extender is locked in place and secure before putting to use.

ADJUSTING LENGTH OF FOOTREST/LEGREST

NOTE: When caregiver is adjusting length of footrests or legrests, user should be sitting in wheelchair with feet on the footplates.

1. Flip up the locking clamp around the support assembly (Fig. D).

2. Raise or lower the footplate to desired position by pressing thumb and index fingers on the two push pins just above the locking clamp. Front rigging should not be so low that it causes uncomfortable pressure to thighs at front of seat upholstery.

WARNING: Footrests should not be lower than 2 ½" from the ground to ensure proper clearance.

3. Once the desired length is found, ensure that both push pins are securely locked in place. An audible click will be heard when they lock in. Flip the locking clamp down into the locked position.
4. Repeat steps 1-3 for the opposite side.

RAISING/LOWERING THE ELEVATING LEGREST

ASSEMBLY (WC4DX1616CE, WC4DX1816CE, WC4DX2016CE, WC4DX2216CE)

1. To raise the elevating legrest, the assistant or user should grab hold of the support tube and raise elevating legrest until desired height is obtained.
2. To lower, support user's leg with one hand. Push release lever downward with one

hand. Gently, lower user's leg down and rest against the legrest (Fig. E).

ADJUSTING HEIGHT OF ARMS

To adjust the height of the armrest, simply push the silver button located on the outside of the armrest, under the armrest pad.

Push or pull armrest to the desired height, lining up into the corresponding hole. Ensure that the adjustment button returns completely back to a locked position. An audible click will be heard (Fig. F).

ADJUSTING FROM DESK LENGTH TO FULL LENGTH:

To adjust from desk length to full length. Push button on outer side of armrest pad. To adjust, either push outward for full length, or inward for desk length. You will hear an audible click once the armrest is in place.

RAISING/LOWERING THE FLIP-BACK ARMS

1. Release the lock-pin and lift the armrest back to get on or get off the wheelchair more convenient.
2. To lower and lock the flip-back arm, push the armrest towards the front of the wheelchair and then downward into the arm socket.

3. Ensure the armrest release lever is in the locked position before use. An audible click will be heard.

HEMI HEIGHT ADJUSTMENT

1. To make a Hemi-Height Adjustment, press down on the release button circled in Figure H.
2. When the quick release button is pressed, the user will be able to pull the axle and rear wheel from the frame to remove.
3. Reinsert the axle and rear wheel into the hemi-height hole of the frame
4. **NOTE:** When sliding the axle into the frame, you will hear an audible click. Figure I shows the quick release axle. The user will hear an audible click when properly in place.

OPERATING INSTRUCTIONS

TRANSFERRING TO AND FROM OTHER SEATS

Position the wheelchair as close as possible alongside the seat to which you are transferring, with the front casters pointing parallel to it. Flip up the armrest. Engage both wheel locks. Shift body weight into seat with transfer. **DO NOT** stand on or apply weight to footrests when transferring to and from the wheelchair. During independent transfer, little or no seat platform will be beneath you. Use a transfer board, if possible.

NOTE: Consult with your healthcare provider before performing this activity independently to ensure you have adequate mobility and upper body strength.

USING WHEEL LOCKS

Engage both wheel locks before getting in or out of chair, leaning forward in chair, or while in an elevator or wheelchair lift. Proper adjustment of wheel lock is at least 1/8" when locked. Engage wheel locks by pushing the handles completely forward. Test for correct locking action before actual use.

STABILITY

The wheelchair is designed to be safe and stable as long as the user's center of gravity is maintained while going about normal activities. Care must be taken while performing activities that shift your center of gravity while in the chair. **NEVER** reach so far that you have to shift in the seat or bend over out of the chair. When you must reach for objects that require you to lean, be certain front casters are pointing in a forward position that extends the wheel base and aid in stabilizing the chair.

RAMPS

Be sure of your own capabilities, limitations and strength before attempting to use a wheelchair ramp without assistance. Correct techniques on ramp negotiation should be demonstrated by a qualified professional. Assistance is highly recommended when going up or down inclines greater than 10°.

If you must stop on an incline, avoid sudden abrupt movement when you resume movement. Sudden movement could cause instability. Attempting to use the wheel locks could result in accidental locking, tipping to one side, or cause a sudden stop. Avoid changing direction when going down an incline, as this could result in instability and tipping over.

NOTE: ALWAYS use the hand rims for self-propulsion. **DO NOT** grip the rear wheels themselves, as this may cause injury to user or excess wear to tires.

CLEANING AND MAINTENANCE

GENERAL CARE

Proper care and maintenance is important to keep your wheelchair in good working condition and avoid premature wear, damage and injury.

Clean your chair frequently, at least once a week, using a clean, dry, soft cloth. **DO NOT** use harsh abrasive materials when cleaning. Dry the chair immediately if exposed to moisture.

WHEELS

Large rear wheels should be inspected at least once a month to ensure proper tightness. If wheel wobbles or is loose, release lock nut and tighten axle bolt slightly.

Check the wheels and casters for rips, wear or cracks regularly. Immediately replace if any of these imperfections exist.

DRIVE WHEEL ADJUSTMENT: Inspect the wheels and quick release axle shafts at least once a month. Correct any side play by tightening the lock nut on the axle shaft just enough to remove the side play but still allowing the wheel to turn freely. Check to see that the axle locks firmly into the axle receiver. A light machine oil should be used to lubricate the axle shaft. Broken spokes should be replaced immediately.

TIRES

Solid polyurethane tires need minimal maintenance, but should be cleaned occasionally with a damp cloth. Replace as needed or immediately if tires become severely worn or start cracking.

FRONT CASTERS

Casters should be checked periodically and lubricated occasionally. Lubricate wheel bearings by removing wheel from fork. Reassemble fork and wheel by tightening nuts until no play remains, but wheels roll freely. If nut is too loose, caster will shimmy. If it is too tight, chair will be difficult to steer.

SEAT UPHOLSTERY

Proper care and maintenance is important to keep your wheelchair in good working condition and avoid premature wear, damage and injury.

Replace worn or damaged upholstery immediately to reduce the risk of injury.

GENERAL CARE

Clean your chair frequently, at least once a week, using a clean, dry, soft cloth. **DO NOT** use harsh abrasive materials when cleaning. Dry the chair immediately if exposed to moisture.

HAND GRIPS

Use only a clean towel dampened with cool water when cleaning hand grips. Confirm that the grips are dry before using. **NEVER** use ammonia-based cleaning solutions or soap on hand grips. This may cause the handgrips to loosen or slide off the frame.

WHEEL LOCKS

Ensure that the wheel locks are adjusted correctly.

We recommend that the chair has at least one full service from an authorized dealer once a year. If you notice any irregular aspect of your chair and/or any of the items are loose worn, bent or distorted, immediately have them checked and/or repaired by your nearest authorized dealer for assistance. They will advise if the chair should be returned for repairs. If the chair is to be returned, they will advise on the availability of replacement units, and the required method of packing and shipping. Frequent maintenance and servicing will improve performance and extend wheelchair life.

CLEANING

Proper care and maintenance is important to keep your wheelchair in good working condition and avoid premature wear, damage and injury.

Clean your chair frequently, at least once a week, using a clean, dry, soft cloth. Do not use harsh abrasive materials when cleaning. Dry the chair immediately if exposed to moisture.

STORAGE AND TRANSPORT

To properly store and transport this wheelchair, pull up on the seat upholstery to fold the wheelchair together. Ensure that the armrests are close together and folded completely. If possible, we recommend laying the wheelchair on its side when transferring to limit the wheelchair's movement.

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LIMITED LIFETIME WARRANTY

This warranty is solely for the benefit of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. Your ProBasics® brand product is warranted to be free of defects in materials and workmanship, has a Limited Lifetime Warranty on the frame and a 12-month warranty from date of purchase on non-durable components, including but not limited to rubber accessories, casters, grips and upholstery, which are subject to normal wear and need periodic replacement. Compass Health Brands warrants the side frames and crossbraces for the lifetime of the original consumer purchaser. If within such warranty period, the product proves defective, the product will be repaired or replaced, at the option of Compass Health Brands.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear.

This warranty shall not apply to serial numbered products if the serial number has been removed or defaced, or to defects or damages caused by unauthorized parts, attachments or repairs, negligence, accident, or any circumstance beyond Compass Health Brands' control, such as improper operation, maintenance or storage.

If you have a question about your Compass Health Brands product or this warranty, please contact an authorized Compass Health Brands Dealer.

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WC4DX_UG_04_240510 ©2024