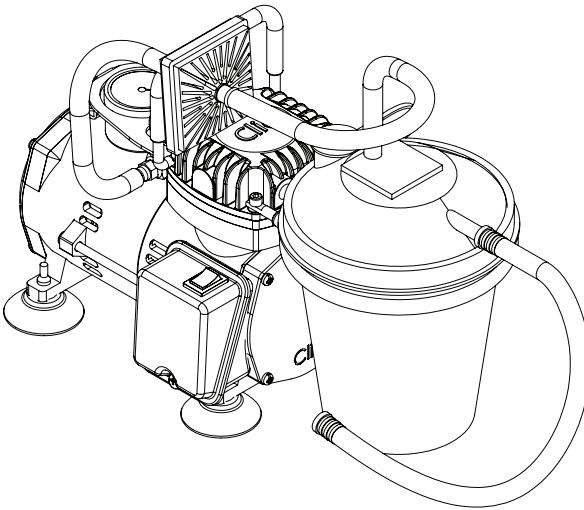
# ROSCOE ASP-ROS

## HEAVY DUTY ASPIRATOR

Manual

MODEL: ASP-ROS

Read this manual before operating the aspirator.  
Save these instructions for future reference.

# TABLE OF CONTENTS

	Page
1. Safety Information . . . . .	3
2. AC Power Source . . . . .	4
3. Electrical . . . . .	4 – 5
4. Introduction . . . . .	5
5. Illustration of Important Parts . . . . .	6
6. Set-Up and Operation . . . . .	7 – 8
7. Cleaning Instructions . . . . .	8 – 9
8. Maintenance . . . . .	9 – 11
9. Specifications / Classifications . . . . .	10
10. Troubleshooting . . . . .	11
11. Warranty . . . . .	12

 <b>CAUTION</b>	Federal law restricts this device to sale by or on order of a physician, or any other practitioner licensed by the law of the State in which he practices to use or order the use of this device.
 <b>NOTE</b>	Check ALL parts for shipping damage. If shipping damage is noted, <b>DO NOT</b> use. Contact medical equipment provider for further instruction.

# **SAFETY INFORMATION**

## **1. SAFETY INFORMATION**

When using electrical products, especially when children are present, basic safety precautions should always be followed. Read all instructions before using. Important information is highlighted by these terms:

 <b>DANGER</b>	Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.
 <b>WARNING</b>	Indicates a potentially hazardous situation which, if not avoided, could result in serious injury or equipment damage.
 <b>CAUTION</b>	Indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury to the user or patient or damage to the device or other property.
 <b>NOTE</b>	Information to which you should pay special attention.

**DO NOT ATTEMPT TO OPERATE THIS PRODUCT WITHOUT  
READING ALL INSTRUCTIONS CAREFULLY.  
SAVE THIS MANUAL FOR FUTURE REFERENCE.**

 <b>DANGER</b>	This medical device is not intended to be used as a life-sustaining/supporting device.
 <b>DANGER</b>	To reduce the risk of electrocution: <ul style="list-style-type: none"> <li>• Do not use while bathing.</li> <li>• Do not place or store product where it can fall or be pulled into a tub or sink.</li> <li>• Do not place in or drop into water or other liquid.</li> <li>• Do not reach for a product that has fallen into water. Unplug immediately.</li> </ul>
 <b>WARNING</b>	To reduce the risk of burns, electrocution, fire or injury to persons: <ul style="list-style-type: none"> <li>• Close supervision is necessary when this product is used by, on, or near children or invalids.</li> <li>• Use this product only for its intended use as described in this guide.</li> <li>• Never operate this product if: <ol style="list-style-type: none"> <li>a. It has a damaged power cord or plug.</li> <li>b. It is not working properly.</li> <li>c. It has been dropped or damaged.</li> <li>d. It has been dropped into water.</li> </ol> </li> <li>• Discontinue use immediately and return the product to an authorized medical equipment service center for examination and repair.</li> <li>• Keep the power cord away from heated surfaces.</li> <li>• Never use while drowsy or asleep.</li> </ul>

 <b>SPECIFIC WARNINGS</b>	<p><b>TO REDUCE THE RISK OF ELECTRICAL SHOCK OR INJURY TO PERSONS, DO NOT DISASSEMBLE THIS UNIT. DISASSEMBLY OR ATTEMPTED REPAIRS IF ACCOMPLISHED INCORRECTLY CAN CREATE ELECTRICAL SHOCK HAZARD. ONLY QUALIFIED TECHNICIANS SHOULD PERFORM SERVICING OF THIS SUCTION UNIT.</b></p>
 <b>DANGER</b>	<p>The Heavy Duty Suction Pump is a vacuum suction device designed for the collection of nonflammable fluid materials in medical applications only. Improper use during medical applications can cause injury or death.</p> <p>For all medical applications:</p> <ul style="list-style-type: none"> <li>• All suctioning should be done in strict accordance with appropriate procedures that have been established by a licensed medical authority.</li> <li>• Some attachment accessories may not fit the tubing supplied. All attachment accessories should be checked prior to use to assure proper fit.</li> </ul>

## 2. AC POWER SOURCE

 <b>CAUTION</b>	<p>Item # ASP-ROS is used with a power supply allowing operation on AC voltage (AC 110-120 V, 60 Hz). The correct power source must be used to connect to adaptable wall power.</p>
----------------------------------------------------------------------------------------------------	-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------

## 3. ELECTRICAL COMPRESSOR

If the overload protector shuts off the motor frequently, you may have a low voltage situation. Low voltage can also be suspected when:

- The motor does not get up to full power or speed.
- Other motor operated appliances fail to operate properly. Too many motor operated appliances on same circuit.

 <b>NOTE</b>	<p>The motor of this compressor has a thermal overload protector. If the motor overheats, the overload protector will shut off the motor. If this occurs, turn the <b>ON/OFF</b> switch to the <b>OFF</b> position and allow the motor to cool (approximately 30 minutes). Restart compressor. If the compressor fails to start, wait an additional 5 minutes before attempting to restart. If this fails, inspect the fuses on your suction unit. If the unit still does not restart, call your medical equipment provider immediately.</p>
----------------------------------------------------------------------------------------------------	----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------

 <p><b>WARNING</b></p>	<p>Proper operation of this product is necessary to avoid physical harm. Improper use can cause personal injury and/or tissue damage. This product should not be used unless the operator has been instructed by a physician or other qualified health care professional.</p> <ol style="list-style-type: none"> <li>1 Familiarize yourself with this unit.</li> <li>2 Place unit on table and push down firmly so that suction feet hold unit securely in place.</li> <li>3 Before plugging power cord into electrical outlet, check if the <b>ON/OFF</b> switch is in the <b>OFF</b> position.</li> </ol>
 <p><b>NOTE</b></p>	<p>Before attempting to start a cold unit, let the unit reach room temperature in a heated area.</p>

## **INTRODUCTION**

Your Heavy Duty Suction Pump is a professional medical suctioning device that produces a maximum vacuum of 675 mmHg. The unit is equipped with an 800cc collection canister and has been designed for convenient and reliable usage.

### Features:

- Microbial filter helps prevent contamination.
- High quality vacuum regulator and anti-vibration vacuum gauge setting.
- Oil-less, permanent lubrication.
- Collection bottle with overflow protection.
- Superior performance and high reliability for continuous operation design.
- Easy carry and easy cleaning.

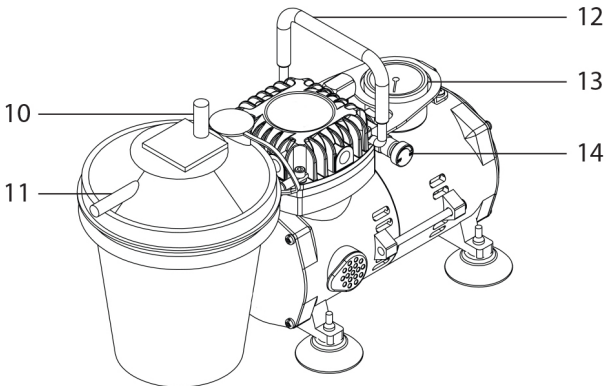
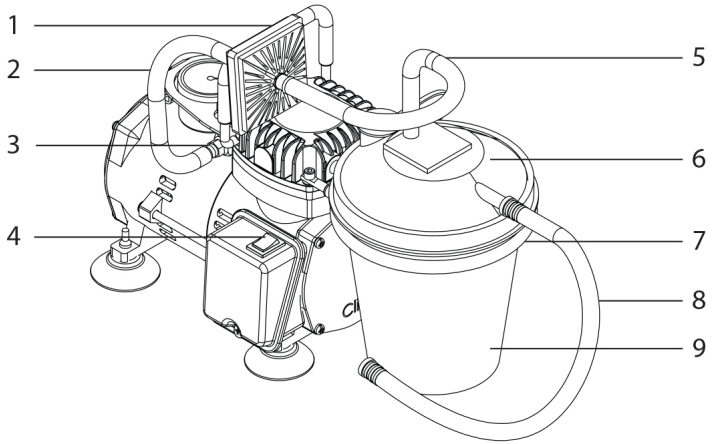
Following the recommended operating and maintenance procedures outlined in this user manual will maximize the life of this product. Refer to the diagrams on page 6.

 <p><b>NOTE</b></p>	<p>This unit complies with electromagnetic compatibility standards as defined in the declaration of conformity.</p>
--------------------------------------------------------------------------------------------------------	---------------------------------------------------------------------------------------------------------------------

# ILLUSTRATION OF PARTS

## ITEM #ASP-ROS

1. Inline filter
2. Long intermediate tubing
3. Inlet
4. Power switch
5. Short intermediate tubing
6. Collection container lid
7. Collection container holder
8. Suction tubing
9. Collection container
10. Vacuum port
11. Patient port
12. Handle
13. Vacuum gauge
14. Vacuum adjustment knob


# SET UP & OPERATION

1. Connect either end of the short intermediate tubing to the vacuum port on top of the collection bottle lid. Then connect the other end to the fluid side of the filter.
2. Connect one end of the long intermediate tubing to the other end of the filter. Attach the other end to the pressure adjusting inlet.
3. The patient tubing should be connected to collection bottle at the outlet labeled "patient". Make sure that all connections are secure and without leaks before operating.

## **HOW TO OPERATE YOUR SUCTION MACHINE:**

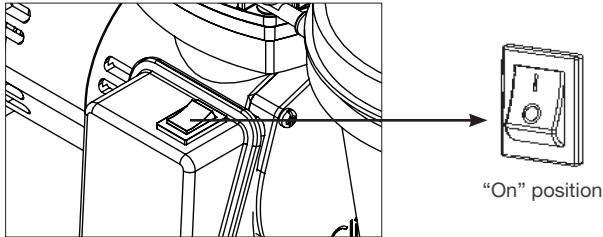
1. Before use, check the specification label on the bottom of the suction unit to ensure that the voltage and current indicated on the unit correspond with the voltage and current available. Make sure the power switch located on the side of the unit is in the "Off" position before connecting the unit to a power source.
2. Switch the power switch to "I" position (On). The unit will start running.

**WARNING**

**NEVER** exceed the recommended maximum vacuum pressure.

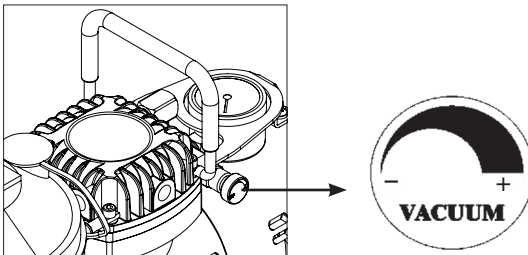3. Completely cover the open end of the patient suction tubing, usually with thumb. Turn the vacuum adjustment knob (clockwise to increase vacuum pressure, counter-clockwise to reduce) until the pressure indicated by the gauge equals the maximum pressure recommended by a trained healthcare professional.

4. Place the patient tubing to the appropriate position of patient and start to suck sputum aspirate.
5. When the sputum reaches the safe full level of collection (800 cc), the suctioning will stop sucking automatically (float shut off).
6. Switch the power switch to “O” position (Off), shut off the suction motor.
7. Remove the sputum from collection bottle after the motor stops turning completely. (Refer to the “Cleaning” section below for the instructions on how to clean the collection bottle.)
8. Turn on the unit and start to run the suction unit again.

 <b>NOTE</b>	<ol style="list-style-type: none"> <li>1. If the unit sustains a severe drop, validity of the gauge must be checked. Inspect if the float shut-off is still functioning properly when the liquid in the collection reaches the safe full level.</li> <li>2. Always transport unit with the vacuum adjust knob rotated fully clockwise in case unit is dropped.</li> </ol>
-----------------------------------------------------------------------------------------------	---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------

 <b>CAUTION</b>	<ol style="list-style-type: none"> <li>1. When automatic float shut-off is activated, contents of the collection bottle should be emptied. Further suctioning could cause damage to the vacuum pump.</li> <li>2. Should fluid be aspirated back into the unit, discontinue use immediately and send your aspirator to an authorized service center for a detailed inspection as the vacuum pump may have been damaged.</li> </ol>
--------------------------------------------------------------------------------------------------	-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------

 <b>WARNING</b>	<p>If the device has been operating continuously for 30 minutes or more, turn off power and allow the device to cool for at least 30 minutes before restarting.</p>
--------------------------------------------------------------------------------------------------	---------------------------------------------------------------------------------------------------------------------------------------------------------------------

## CLEANING

### **CLEAN COLLECTION BOTTLE**

1. Turn off the aspirator and allow vacuum to drop, then disconnect AC power source.
2. With the collection bottle still in the holder, remove the collection bottle lid. The bottle can now be taken out of the holder to be emptied.

 <b>NOTE</b>	<p>Collection bottle should be emptied after each use.</p>
-------------------------------------------------------------------------------------------------	------------------------------------------------------------

3. The collection bottle should be thoroughly cleaned after each use by one of the following three methods:
  - a. Wash in a hot water/dishwashing detergent solution and rinse with clean, hot tap water. Then wash in one part vinegar to three parts hot water solution. Rinse with hot tap water and air dry.
  - b. Wash with rubbing alcohol and air dry.

- c. Wash with a commercial (bacterial-germicidal) disinfectant. Follow disinfectant manufacturer's recommended instructions and dilution rates carefully.

### **CLEAN SUCTION UNIT AND COLLECTION BOTTLE LID**

1. Disconnect the suction unit from all external power sources.
2. Wipe the suction unit housing and collection bottle lid with clean cloth and any commercial (bacterial-germicidal) disinfectant. Follow disinfectant manufacturer's recommended instructions and dilution rates carefully.

 <b>CAUTION</b>	<b>DO NOT</b> submerge in water as this will result in severe damage to the vacuum pump. <b>DO NOT</b> clean with powder-type cleaning agents or soap.
--------------------------------------------------------------------------------------------------	--------------------------------------------------------------------------------------------------------------------------------------------------------

### **CLEAN TUBING**

1. Disconnect the tubing from the unit.
2. Tubing should be rinsed thoroughly after every use by running hot tap water through it followed by a solution of one part vinegar to three parts hot water.
3. Rinse with hot tap water and air dry.
4. Keep the outer surface of the tubing clean by wiping with a clean, damp cloth.

 <b>WARNING</b>	The waste in collection bottle may cause respiratory infection, which must be handled according to medical waste disposing regulation. Operator must wear gloves and medical-class mask before cleaning the collection tubing and handling the collection bottle and/or accessories.
-------------------------------------------------------------------------------------------------	--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------

## **MAINTENANCE**

### **MAINTENANCE**

Before each use, inspect power cord for fraying/damage and all accessories (collection canister, tubing and microbial filter) for cracks/leaks.

 <b>DANGER</b>	<ul style="list-style-type: none"><li>• Electric shock hazard</li><li>• Do not attempt to open or remove the suction cabinet. If a service is required, return your suction to an authorized service center. Opening or tampering with the unit will void warranty.</li></ul>
--------------------------------------------------------------------------------------------------	-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------

### **CHANGING THE FILTER**

1. Filter should be replaced every two months. If overflow occurs, change the filter immediately.
2. Remove filter by disconnecting it from tubing connected to pressure adjusting valve and tubing connected to collection lid assembly.

3. Replace with a new filter and reconnect to the short and long intermediate tubing (see “Setup & Operation” on pg. 7). Additional filter can be purchased from an authorized dealer.

## **SPECIFICATIONS/CLASSIFICATIONS**

Dimensions	13"L x 5.9"W x 8.3"H / 33 x 15 x 21 cm
Weight	7.7 lbs / 3.5 kg
Vacuum	0 - 675 mmHg
Max Working Airflow	~15-17 LPM
Voltage Requirements	AC 110-120V / 60 Hz
Collection Canister Capacity	800 mL (cc)
Power Cord Length	6' / 185 cm
Operating Temp	32° - 104°F / 0° - 40°C
Operating Relative Humidity	0-95% RH
Storage and Transport Temp	40° - 158°F / -40°-70°C
Storage and Transport Humidity	0-95% RH
Operating Atmospheric Pressure	10.2 ~15.4 psi (70 ~106 kpa)
Storage Temperature	14° ~ 158° F (-10° ~ 70°C)

## **MAINTENANCE**

 <b>NOTE</b>	Do not substitute any other material for the filter. Improper substitution may lead to contamination or poor performance.
-----------------------------------------------------------------------------------------------	---------------------------------------------------------------------------------------------------------------------------

### **COLLECTION CANISTER MAINTENANCE**

1. Discard contents at least daily
2. Wash canister with hot water and a mild dish detergent.
3. Every 1-3 days, soak canister in a 1:3 mixture of white vinegar and water for 30 minutes. Rinse well with water.
4. Discard and replace canister as needed, or every three months.

 <b>WARNING</b>	Residues in collection bottle, tubing and filter may cause respiratory infection. Dump the waste in the collection bottle, tubing and filter into a specific waste bin labeled with medical waste.
----------------------------------------------------------------------------------------------------	----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------

## **TROUBLESHOOTING**

Your suction machine contains no user-serviceable parts. If you believe your unit is not working properly, BEFORE YOU RETURN IT TO THE MEDICAL EQUIPMENT PROVIDER WHERE YOU PURCHASED IT, please take a few moments to check for these possible causes:

<b>PROBLEM</b>	<b>ACTION</b>
Unit does not turn on	<ol style="list-style-type: none"><li>1. Check power source and connection</li><li>2. Ensure that the wall outlet is live</li><li>3. Check the power cord for damage</li><li>4. Allow machine to cool for 30 minutes before resuming operation</li><li>5. If the fibrous material inside the inline filter is wet, stained or damaged, replace the filter immediately</li></ol>
Motor runs, no vacuum	<ol style="list-style-type: none"><li>1. Verify tubing connection security</li><li>2. Check for leaks or tubing kinks</li><li>3. Ensure that float shut-off is not activated</li><li>4. Check for leaks and cracks</li></ol>
Low Vacuum	<ol style="list-style-type: none"><li>1. Use vacuum adjustment knob to increase vacuum level</li><li>2. Check system for leaks</li><li>3. Adjust vacuum adjust knob and release</li><li>4. Check tubing port for restriction</li><li>5. Check voltage and current levels on power outlet</li><li>6. Replace canister, tubing and/or filter if damaged</li></ol>

## **LIMITED WARRANTY**

Compass Health Brands offers a limited two (2) year warranty on the compressor unit and six (6) months on accessories. We warrant that the compressor unit and its parts will be free from defects in workmanship and materials for a period of two (2) years from the date of original purchase and thirty (30) days on tubing and accessories. This warranty does not extend to failures resulting from accident, misuse, abuse, alteration, use of unauthorized service, parts or cleaning solutions, or failure to comply with the User's Manual. If the warranted equipment should fail during the warranty period, Compass Health Brands, at our option, will repair or replace parts at issue, provided the claim is bona fide. It shall be the responsibility of the purchaser to pack and return the equipment in a manner to avoid shipping damage. The shipment must be prepaid to us. This warranty covers the cost of labor incurred in the removal or replacement of the warranted component parts at us only. please contact an authorized Compass Health Brands dealer.

## **SERVICE AND REPAIR**

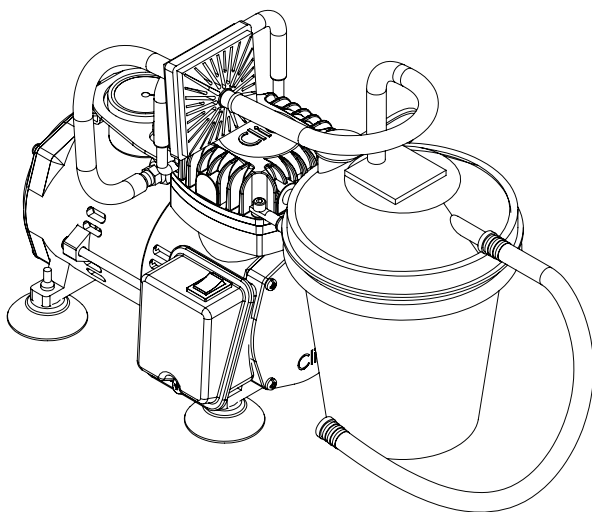
If you should have any questions about this device, please contact an authorized Compass Health Brands dealer. DO NOT open or disassemble the compressor unit at any time. This will void the warranty. If the unit is malfunctioning, discontinue use immediately and return to the original provider for repair or replacement. Please dispose of this device in accordance with local, municipal, state, and federal law of electronic devices

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42-ASP-ROS\_00 ©2020
Compass Health Brands Corp.