Bariatric Drop-Arm Commode

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*The height adjustable legs are included with item #BSBDAC only.

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Congratulations on purchasing a quality ProBasics® Bariatric Drop-arm Commode. Our drop-arm commodes have a versatile design that allow for safe and easy lateral transfers. Please be sure to read these instructions and warnings thoroughly and follow the instructions of your healthcare provider to maximize the commode's benefits and your safety.

DO NOT ATTEMPT TO ADJUST OR OPERATE THIS DROP-ARM COMMODE WITHOUT READING ALL INSTRUCTIONS CAREFULLY.

ASSEMBLY INSTRUCTIONS

Remove the bariatric drop-arm commode components from all packaging including tape, cardboard and plastic. Check all parts for damage before use. If any parts are missing or damaged, **DO NOT** attempt to use the commode. If you do not understand the manual, contact your dealer for a complete explanation of the instructions, cautions and warnings. Otherwise injury may result.

NOTE: DO NOT use excessive force when installing any parts.

SETUP INSTRUCTIONS

1. Remove frame, seat, lid, and pail from carton. Allow the seat and lid to reach room temperature if exposed to cold. This will help prevent the seat and lid clamps from cracking while being assembled onto commode front and back frame rails.
2. Align the toilet seat clamps onto the front and back frame rails. Push down on platform seat with both hands on all four corners until clamps snap firmly into place.
3. To remove the seat, grasp armrest in locked position for leverage with one hand, and pull up platform seat corner attached to the front frame rail with other hand. Repeat on the opposite corner on back frame rail, using the back support for leverage.
4. From the front of the frame, slide the bucket (without lid) under the platform seat.
5. Attach the back support by depressing the brass push pin and pushing until both are locked securely in place. An audible click will be heard.
6. The commode is now ready for use.

ADJUST COMMUNE HEIGHT (BSBDAC ONLY)

1. Depress the height adjustment button and slide the lower leg upward or downward until the height adjustment button is fully engaged in the desired hole.
2. Repeat for other three leg attachments.

CAUTION: Ensure that all four commode legs are adjusted to the same height, and that all height adjustment buttons protrude fully through adjustment holes before use.

NOTE: The 26" and 36" Drop-arm Commode models are **NOT** height adjustable.

USE AS A BEDSIDE COMMODE

1. Set commode to desired height (see "Adjust Commode Height" section - for BSBDAC model only).
2. Slide commode pail under platform seat into pail support bracket. Ensure that pail is properly positioned below commode opening. Commode is now ready for use.
3. When user is finished using commode, place commode pail lid over commode pail to contain odors.
4. Remove pail and lid simultaneously to empty pail. Clean seat and pail with disinfectant after each use.

DROP-ARM USAGE

1. To lower drop-arms, push in latch and gently move arm downward to rest alongside the frame.
2. To raise drop-arms, gently pull up arms until they are fully seated in latches. An audible click will be heard.

MAINTENANCE

1. Use a mild soap and water solution, rinse thoroughly with clean water, then dry with a soft cloth. Ensure that commode is well rinsed and dried before using.
2. Never use organic solvents, abrasive cleaners or scouring pads on any part of commode.
3. Check fit and tightness of all nuts, bolts and knobs **WEEKLY** to ensure that the commode is stable and safe to use.
4. Check the rubber tips on the legs for rips, wear or if they are missing. Immediately discontinue use and replace any or all parts if any of these imperfections exist.

WARNINGS

DO NOT ATTEMPT TO ADJUST OR OPERATE THE DROP-ARM COMMODE WITHOUT READING ALL INSTRUCTIONS CAREFULLY.

- Maximum weight capacity: 650 lb/295 kg. (BSBDAC), 800 lb/363 kg. (BSB26DAC), 1000 lb/454 kg. (BSB36DAC).
- **DO NOT** use this device if it is not securely installed.
- This product should not be used without instruction from a healthcare professional.
- The platform seat must be securely attached to the frame prior to use. Failure to do so may result in injury or damage to the commode.
- Ensure the height adjusting snap buttons fully protrude through the same respective hole of each leg extension. This will ensure the leg extensions are locked in position and an even height is achieved (BSBDAC only)
- All four leg extensions with rubber tips must touch the floor at all times.
- Ensure drop-arms are secured and locked before use.
- Users with limited physical strength should be supervised or assisted while using commode.
- Always use caution when floor is wet. Ensure armrest handles are dry before use.
- Use the armrest for assistance only. **DO NOT** use the armrests to support full body weight.
- Weight should be evenly distributed when using the commode. **DO NOT** lean or reach for objects while on the commode.
- **DO NOT** stand on the commode seat or frame.
- Inspect all parts for shipping damage before assembly. If there is shipping damage — **DO NOT USE**. Contact dealer/carrier for further instructions.
- Users with limited physical capabilities should be supervised or assisted in commode areas even when using support rails.
- Consult a healthcare professional for advice and instructions specific to your needs before use.
- Compass Health Brands assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.

USE CAUTION. FAILURE TO USE COMMON SENSE AND HEED THE ABOVE WARNINGS FURTHER INCREASES RISK OF SERIOUS INJURY. USE AT YOUR OWN RISK AND WITH APPROPRIATE AND SERIOUS ATTENTION TO SAFE OPERATION.

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LIMITED LIFETIME WARRANTY

This warranty is solely for the benefit of the original consumer purchaser.

This device was built to exacting standards and was carefully inspected prior to shipment. Your ProBasics® brand product is warranted to be free of defects in materials and workmanship, has a Limited Lifetime Warranty on the frame and a one year warranty from date of purchase on all other parts, which are subject to normal wear and need periodic replacement. If within such warranty period, the product proves defective, the product will be repaired or replaced, at the option of Compass Health Brands. This warranty does not cover product failures due to normal wear and tear or device misuse.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty provides only for replacement of defective parts and does not cover shipping or labor charges. Compass Health Brands' sole obligation and your exclusive remedy under this warranty is limited to such repair and/or replacement.

This warranty shall not apply to serial numbered products if the serial number has been removed or defaced or to defects or damages caused by unauthorized parts, attachments or repairs, negligence, accident, circumstances beyond Compass Health Brands' control, or improper operation, maintenance or storage.

If you have a question about your Compass Health Brands product or this warranty, please contact an authorized Compass Health Brands Dealer.

COMPASSHEALTH

6753 Engle Road

Middleburg Heights, Ohio 44130

Tel.: (800) 376-7263

www.compasshealthbrands.com

42-BSBDAC_BSB26DAC_BSB36DAC_BariatricDrop-

ArmCommode_00 ©2017