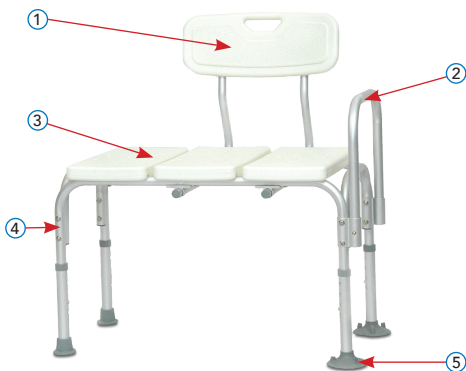
# Transfer Bench

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Congratulations on purchasing a quality ProBasics® Transfer Bench. Our transfer benches offer stability, comfort and easy installation. Please be sure to read these instructions and warnings thoroughly and follow the instructions of your healthcare provider to maximize the transfer bench's benefits and your safety.

**DO NOT ATTEMPT TO ADJUST OR OPERATE THIS TRANSFER BENCH WITHOUT READING ALL INSTRUCTIONS CAREFULLY.**

## ASSEMBLY INSTRUCTIONS

Remove the transfer bench and any components from all packaging including tape, cardboard and plastic. Check all parts for damage before use. If any parts are missing or damaged, **DO NOT** attempt to use the transfer bench. If you do not understand the manual, contact your dealer for a complete explanation of the instructions, cautions and warnings. Otherwise injury may result.

**NOTE: DO NOT** use excessive force when installing any parts.

### SETUP INSTRUCTIONS

1. Remove frame, back support, support arm and height adjustable legs from box.
2. Insert support arm into frame until support arm buttons protrude fully through channel guide holes. An audible click will be heard.
3. Place frame upside down on working surface.
4. Insert back support into base by depressing push pins and sliding supports through both receptacles. Back support must go through both sets of receptacles to become secure. An audible click will be heard.
5. Attach suction cup legs to frame on same side as support arm. Attach remaining two legs to the opposite side of frame.
6. Adjust seat height as directed in "Height Adjustment" section.

**NOTE:** When the transfer bench is in the tub, it should face the faucets with the support arm on inside of bath. If not, repeat step 4, and reverse the position.

**NOTE:** When removing the transfer bench from the tub, always release the suction cups by pinching the two tabs together.

## HEIGHT ADJUSTMENT

1. Depress the height adjustment button and slide the lower leg upward or downward until the height adjustment button is fully engaged in the desired setting.
2. Repeat for other three legs.

**NOTE:** Legs inside the tub may be adjusted to a different height than the legs outside the tub. However, the legs inside the tub should be adjusted to the same height, and the legs outside the tub should be adjusted to the same height.

## USING YOUR TRANSFER BENCH

### **From a seated position (such as a wheelchair):**

1. Place your wheelchair next to the bench, facing the front of the bathtub.
2. Engage wheelchair locks.
3. Transfer your torso to the transfer bench first, then slide bottom to the transfer seat.
4. Lift legs into the tub, one at a time, and moving slowly to prevent injury.
5. After bathing, slowly lift legs out of the tub one at a time, then your torso.

### **From a standing position:**

1. Sit on the side of the transfer bench that is outside of the tub.
2. Extend your arms to grab the support arm of the bench and slide over onto the bench. Lift each leg over the tub slowly.
3. Slide body to a comfortable position in the center of bench.
4. After bathing, slowly lift legs out of the tub one at a time, then your torso.

## MAINTENANCE

1. Use a mild soap and water solution, rinse thoroughly with clean water, then dry with a soft cloth. Ensure that commode is well rinsed and dried before using.
2. Never use organic solvents, abrasive cleaners or scouring pads on any part of transfer bench.
3. Check fit and tightness of all nuts, bolts and knobs **WEEKLY** to ensure that the transfer bench is stable and safe to use.

## WARNINGS

### **DO NOT ATTEMPT TO ADJUST OR OPERATE THE TRANSFER BENCH WITHOUT READING ALL INSTRUCTIONS CAREFULLY.**

- Maximum weight capacity: 300 lb/159 kg (BSTB, BSTBP); 500 lb/227kg (BSBTB).
- **DO NOT** use this device if it is not securely installed.
- This product should not be used without instruction from a healthcare professional.
- Use the side support for assistance only. **DO NOT** use side support to support full body weight.
- All four leg extensions with rubber tips/suction cups must touch the floor at all times.
- Ensure **ALL** screws, nuts and/or bolts are tightened and threading is intact on a weekly basis.
- Check all the rubber tips and suction cups on the leg extensions for rips, wear or if they are missing. Immediately replace any or all if any of these imperfections exist.
- Weight should be evenly distributed when using the transfer bench. **DO NOT** lean or reach for objects while seated on the transfer bench.
- **NEVER** stand on the transfer bench.
- It is recommended that the user ease down onto the seat. **DO NOT** sit on the seat with direct force.
- Always use caution when floor or surfaces are wet.
- Inspect all parts for shipping damage before assembly. If there is shipping damage — **DO NOT USE**. Contact dealer/carrier for further instructions.
- Users with limited physical capabilities should be supervised or assisted in shower or tub areas even when using support rails.
- Accessories designed by other manufacturers have not been tested by Compass Health Brands and are not recommended for use with this product.
- Consult a healthcare professional for advice and instructions specific to your needs before use.
- Compass Health Brands assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.

**USE CAUTION. FAILURE TO USE COMMON SENSE AND HEED THE ABOVE WARNINGS FURTHER INCREASES RISK OF SERIOUS INJURY. USE AT YOUR OWN RISK AND WITH APPROPRIATE AND SERIOUS ATTENTION TO SAFE OPERATION.**

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# LIMITED LIFETIME WARRANTY

This warranty is solely for the benefit of the original consumer purchaser.

This device was built to exacting standards and was carefully inspected prior to shipment. Your ProBasics® brand product is warranted to be free of defects in materials and workmanship, has a Limited Lifetime Warranty on the frame and a one year warranty from date of purchase on all other parts, which are subject to normal wear and need periodic replacement. If within such warranty period, the product proves defective, the product will be repaired or replaced, at the option of Compass Health Brands. This warranty does not cover product failures due to normal wear and tear or device misuse.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty provides only for replacement of defective parts and does not cover shipping or labor charges. Compass Health Brands' sole obligation and your exclusive remedy under this warranty is limited to such repair and/or replacement.

This warranty shall not apply to serial numbered products if the serial number has been removed or defaced or to defects or damages caused by unauthorized parts, attachments or repairs, negligence, accident, circumstances beyond Compass Health Brands' control, or improper operation, maintenance or storage.

If you have a question about your Compass Health Brands product or this warranty, please contact an authorized Compass Health Brands Dealer.

**COMPASSHEALTH**

6753 Engle Road  
Middleburg Heights, Ohio 44130  
Tel.: (800) 376-7263  
[www.compasshealthbrands.com](http://www.compasshealthbrands.com)  
42-BSTB\_TransferBench\_00 ©2017