3.75"

Manufactured for: Compass Health Brands Corp.  
6753 Engle Road  
Middleburg Heights, Ohio 44130  
Tel.: (800) 376-7263  
www.compasshealthbrands.com  
Made in China / Hecho en China  
42\_CRA\_CRAT\_CRJ\_Crutches\_00 ©2017

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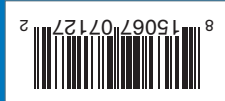
Not Made with Natural Rubber Latex  
Sin látex  
Sans latex

350 lb/158 kg Weight Limit  
Límite de peso: 350 lb (158 kg)  
Limite de poids : 350 lb/158 kg

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3.75"

# Aluminum Crutches

Tool Free Handle & Height Adjustment

For optimal fit and assembly, see instructions below:

1. Height: Adjust height by pressing both push-buttons, rotating foot-piece, and pulling down until desired patient height is obtained. To help in doing this, the foot-piece has a circular ring marked to indicate location of push-button. Rotate foot-piece again until both push buttons snap back into place.
2. Handgrip: Adjust handgrip to fit an approximate 10° bend at patient elbow.
3. Safety Check: With crutches positioned per diagram at the end of the instructions, confirm patient can lean forward comfortably while resting on the underarm supports. Patient should not have to stand uncomfortably upright. Ensure both push-buttons are firmly in place and handgrip nuts are tightened securely.

## CAUTIONS:

- Use only after proper training under the direction of your doctor or physical therapist.
- Replace crutch tips if worn.
- Keep hand grip wing nut tightened at all times.
- Use extra caution on wet and slippery surfaces.
- Limited lifetime warranty on frame, 90 days on all other parts

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