



## Homecare Bed Storage Cart with Shelf - Model: PBHBCART36

Thank you for choosing the ProBasics Bed Storage Cart. It is designed to fit most 36" wide Homecare Beds and is the perfect option for convenient storage and delivery. Use the bed ends to create a simple and compact unit to manage beds.


**Components** - Assembled Shelf

### Cleaning

The shelf may be cleaned with a soft, damp cloth and mild detergent or disinfectant. **DO NOT** use abrasive cleaners or stiff-bristled brushes to avoid scratching the finish on the shelf assembly. **ALWAYS** allow the shelf to fully dry before use.

### Maintenance

Between uses or every 3 months, check that the shelf screws are tightly secured and that the shelf/frame has no deformation due to use.

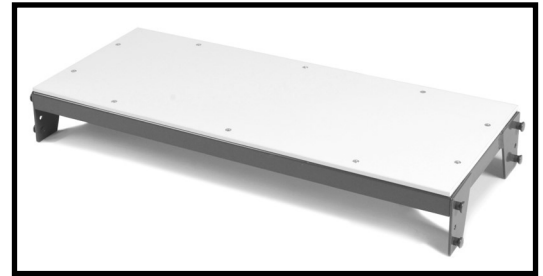

---


### Assembly instructions

- A. Lock shelf in bed connectors on the frame
- B. Use included corner locks to secure bed connectors to frame

A



B

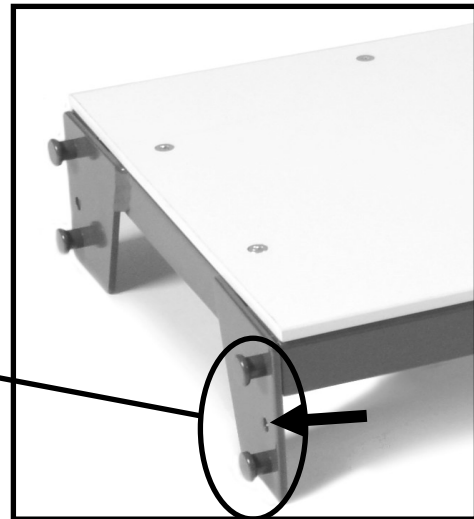

Compass Health Brands  
6753 Engle Road  
Middleburg Heights, OH  
44130

Made in China