

## IMPORTANT NOTICE

The hand brake system fitted on this rollator complies with the manufacturer's quality standards. The hand brake system has been preset (adjusted) to the best performance settings. To better suit you personally, please follow the instructions set out below. You may need to adjust the hand brake systems from time to time as they will wear out with normal use.

## INSTRUCTIONS TO ADJUST THE HAND BRAKE SYSTEM

HOLD FIRMLY IN PLACE.  
DO NOT TURN.

TO TIGHTEN THE HAND  
BRAKE ADJUSTMENT,  
TURN THE NUT CLOCKWISE

TO LOOSEN THE HAND  
BRAKE ADJUSTMENT,  
TURN THE NUT  
COUNTER CLOCKWISE

