# Trigger Release Folding Walker

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## Walker Parts Included:

4 Leg Extenders

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2 Leg Extenders + 2 Leg Extenders w/Wheels

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\* The height adjustable front wheels and glide leg tips are included with Item #'s WKAAWSLB, WKAAWSL, and WKAJWSL

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Congratulations on purchasing a quality ProBasics® Trigger Release Folding Walker. Our walkers are lightweight and convenient in providing stability and balance in your daily activities. Please be sure to read these instructions and warnings thoroughly and follow the instructions of your healthcare provider to maximize the walker's benefits and your safety.

**DO NOT ATTEMPT TO ADJUST OR OPERATE THIS WALKER WITHOUT READING ALL INSTRUCTIONS CAREFULLY.**


## ASSEMBLY INSTRUCTIONS

Remove all packaging including tape, cardboard and plastic before use. Lay out all the pieces and make sure to inspect them prior to assembly. If any parts are missing or damaged, **DO NOT** attempt to assemble the walker.

Contact your distributor immediately.


**NOTE: DO NOT** use excessive force when installing any parts.

1. Hold the folded frame in the upright position (Fig. A).
2. Unfold the two (2) side frame pieces until you hear an audible click from each of the trigger releases (Fig. B). Stand the walker in the upright position. Gently push the side frame pieces towards the center to ensure the trigger releases have securely locked into place.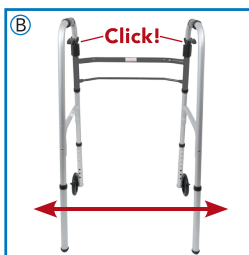

## TRIGGER RELEASE WALKER — WITHOUT WHEELS


**NOTE:** If you have purchased a trigger release walker with 5" wheels, please skip to the next section.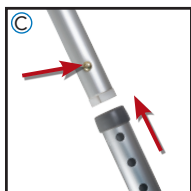


1. Insert one (1) leg into the end of one of the side frame pieces (Fig. C).
2. Insert the leg by depressing the brass push pin and sliding the leg until it is securely in place. When the leg is securely in place, the brass push pin will click into place and extend out of the locking hole.
3. Repeat this step for the remaining three (3) legs.

## TRIGGER RELEASE WALKER — WITH 5" WHEELS

**NOTE:** If you have purchased a trigger release walker without wheels, please read the previous section.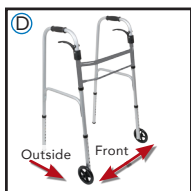

1. When installing the wheel legs on your two button walker, you must make sure they are installed on the front two legs and the wheels are facing outward (Fig. D).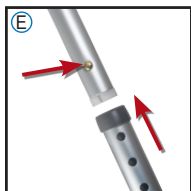


2. Insert one (1) leg tip into the end of one of the back legs of the side frame (Fig. E). Insert the leg tip by depressing the brass push pin and sliding it until it is securely in place. When the leg tip is securely in place, the brass push pin will click into place and extend out of the locking hole. Repeat this step for the other leg tip.

3. Insert one (1) wheel leg into the end of one of the front legs of the side frame (Fig. E). Insert the wheel leg by depressing the brass push pin and sliding it until it is securely in place. When the wheel leg is securely in place, the brass push pin will audibly click into place and extend out of the locking hole. Repeat this step for the other wheel leg.
4. Make sure the two (2) front wheels are on the outside of the walker (Fig. D).

**WARNING: THE WALKER MUST BE LEVEL BEFORE USE. ALL LEG ATTACHMENTS MUST BE ADJUSTED TO THE SAME SETTING TO MAKE THE SIDE FRAMES LEVEL.**

## ADJUSTING THE HEIGHT

Before this trigger release walker can be used, the handle height will need to be adjusted to the user's specific needs. Once the walker has been opened, legs are installed and the sides have been confirmed as locked into place, stand with the middle of your feet even with the walker's back legs.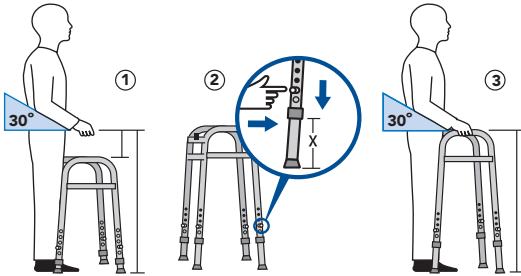

1. With arms relaxed at your side, the handle of the walker should be at wrist height. Or with your hands on the hand-grips, your arms should be flexed  $20^{\circ}$  –  $30^{\circ}$ .
2. Depress the brass push pins and slide each leg until the desired height is reached. Once the desired height has been reached, lock the leg securely in place by sliding the leg until the brass push pin clicks into place and extends out of the locking hole. Repeat this step for the other three legs. The lock holes have been numbered to ensure all four legs match the same height.
3. To be sure the legs are securely locked into place, gently push down on the hand grips or pull on each of the legs.

**CAUTION:** Ensure that all four walker legs are adjusted to the same height and that **ALL** brass push pins are extended through the locking holes before use.


## WALKING WITH THE WALKER

1. Start with the middle of your feet even with the walker's back legs.
2. Holding the hand grips, pick the walker up and place it in front of you at an arms length that feels comfortable. Be sure all four legs return to the floor.
3. Carefully step forward with your weaker leg into the middle of your walker followed by your stronger leg. This will help keep weight off your weaker leg by using your arms for added support.

## FOLDING THE WALKER

This trigger release walker is designed to fold quickly and easily for transportation and storage.

1. Standing behind the walker, grab the trigger lever on the right side and squeeze (Fig. G).
2. Swing the side rail all the way towards the center of the walker (Fig. H).
3. Repeat for left side until completely folded (Fig. I).

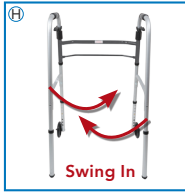## MAINTENANCE

- To clean, use a mild soap and water solution, rinse with clean water on a damp cloth, then dry with a soft cloth.
- Ensure that the walker is thoroughly dried before using.
- **NEVER** use organic solvents, abrasive cleaners or scouring pads on any part of the walker.
- Check all the parts and movable connections before each use to ensure that the walker is safe to use. If deemed unsafe, immediately discontinue use and replace any or all parts in question.
- Check the rubber tips on the height adjustable legs for rips, wear or if they are missing. Immediately discontinue use and replace any or all parts if any of these imperfections exist.
- To ensure the best service from your product, always use ProBasics® replacement parts. Contact your authorized dealer for more information.

## WARNINGS

- **DO NOT ATTEMPT TO ADJUST OR OPERATE THIS WALKER WITHOUT READING ALL INSTRUCTIONS CAREFULLY.**

# WARNINGS

- Maximum weight capacity: 350 lb (159 kg).
- Make sure your walker is adjusted to your correct height.
- **DO NOT** stand on walker.
- Inspect all parts for shipping damage before assembly. If there is shipping damage — **DO NOT USE**. Contact dealer/carrier for further instructions.
- This product should not be used without instruction from a healthcare professional.
- Do not use this walker in pools, showers or other water environments, as this could result in corrosion of parts that result in damage to the walker and possible injury to the user.
- Always avoid unsafe walker conditions, such as soft, slippery or moving surfaces, uneven paths and obstacles.
- Do not exceed the maximum weight capacity intended for this product.
- If you are transferring to or from a wheelchair to use this product, be sure the wheelchair wheels are securely locked.
- This walker is not intended for use on stairs.
- Do not push one side of the walker ahead and then other side without lifting the walker first.
- This walker is **NOT** equipped with rear wheel brakes, **DO NOT** install the optional 5" wheels on the back legs.
- Optional 5" wheels **MUST** be installed on the front legs only with the wheels facing the outside of the walker.
- Make sure that the brass push buttons on all the parts are fully protruding through the locking holes. This will ensure that all connections are locked in position.
- Check the rubber tips on the height adjustable legs for rips, wear or if they are missing. Immediately replace any or all if any of these imperfections exist.
- Accessories designed by other manufacturers have not been tested by Compass Health Brands and are not recommended for use with this product.
- **DO NOT** use the hand grips to fully support the user's body weight.
- Users with limited physical capabilities should be supervised or assisted when using the walker.
- Compass Health Brands assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.

**FAILURE TO FOLLOW THE  
ABOVE INSTRUCTIONS MAY RESULT  
IN SERIOUS INJURIES.**

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## LIMITED LIFETIME WARRANTY

This device was built to precise standards and was carefully inspected prior to shipment. Your ProBasics® brand product is warranted to be free of defects in materials and workmanship, has a Limited Lifetime Warranty on the frame and a one year warranty from date of purchase on all other parts, which are subject to normal wear and need periodic replacement. If within such warranty period, the product proves defective, the product will be repaired or replaced, at the option of Compass Health Brands. This warranty does not cover product failures due to normal wear and tear or device misuse.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty provides only for replacement of defective parts and does not cover shipping or labor charges. Compass Health Brands' sole obligation and your exclusive remedy under this warranty is limited to such repair and/or replacement.

This warranty shall not apply to serial numbered products if the serial number has been removed or defaced, or to defects or damages caused by unauthorized parts, attachments or repairs, negligence, accident, circumstances beyond Compass Health Brands' control, or improper operation, maintenance or storage.

If you have a question about your Compass Health Brands product or this warranty, please contact an authorized Compass Health Brands Dealer.

**COMPASSHEALTH**

6753 Engle Road  
Middleburg Heights, Ohio 44130  
Tel.: (800) 376-7263

[www.compasshealthbrands.com](http://www.compasshealthbrands.com)

42\_ WKAANSL\_TriggerReleaseFoldingWalker\_00 ©2017